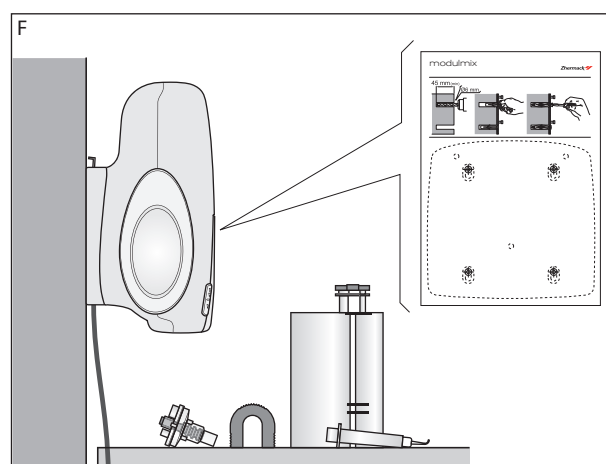
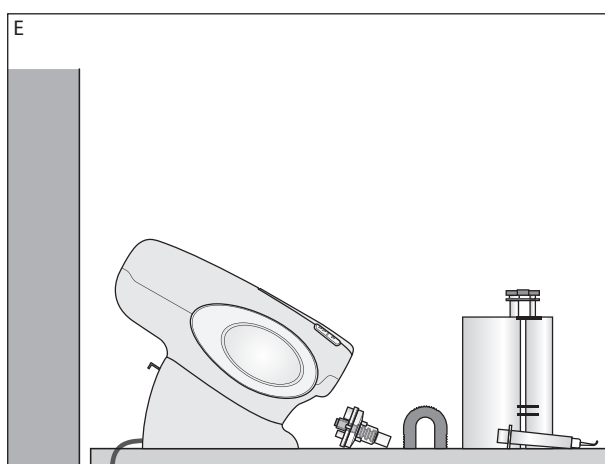
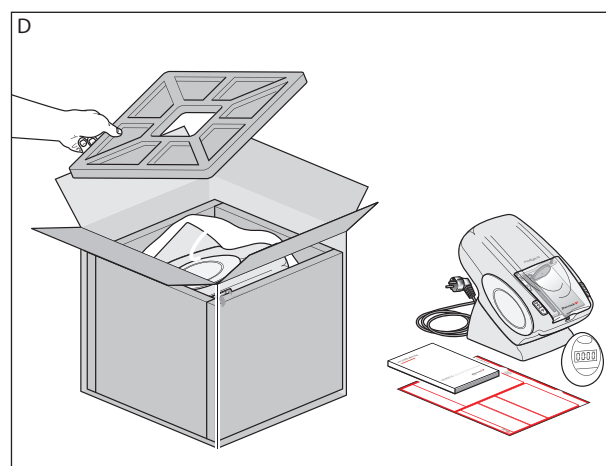
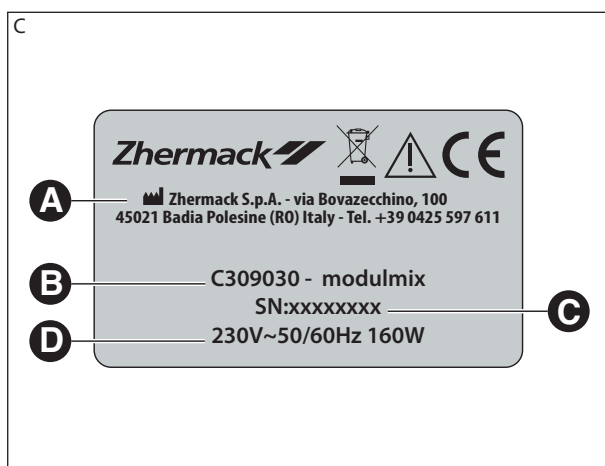
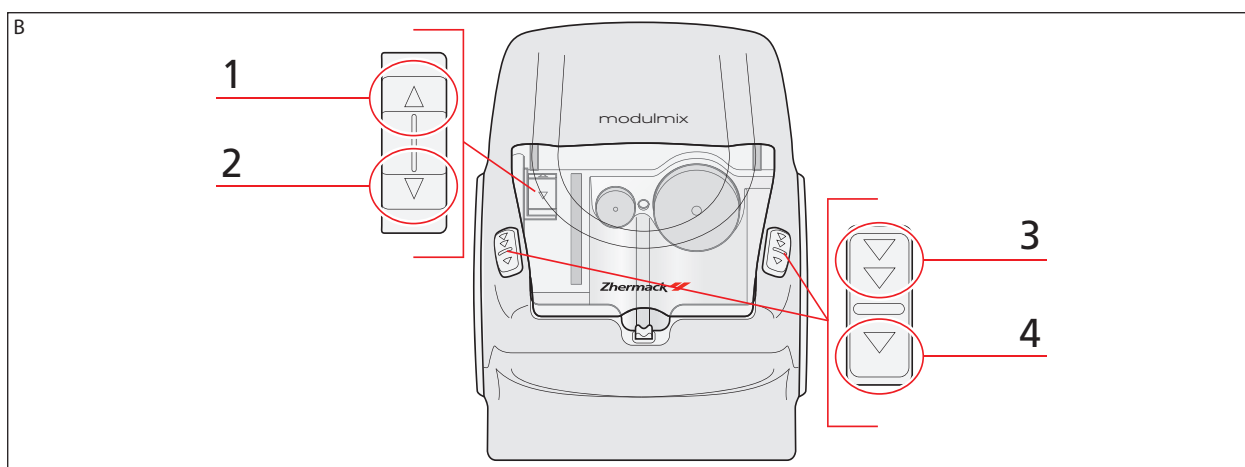
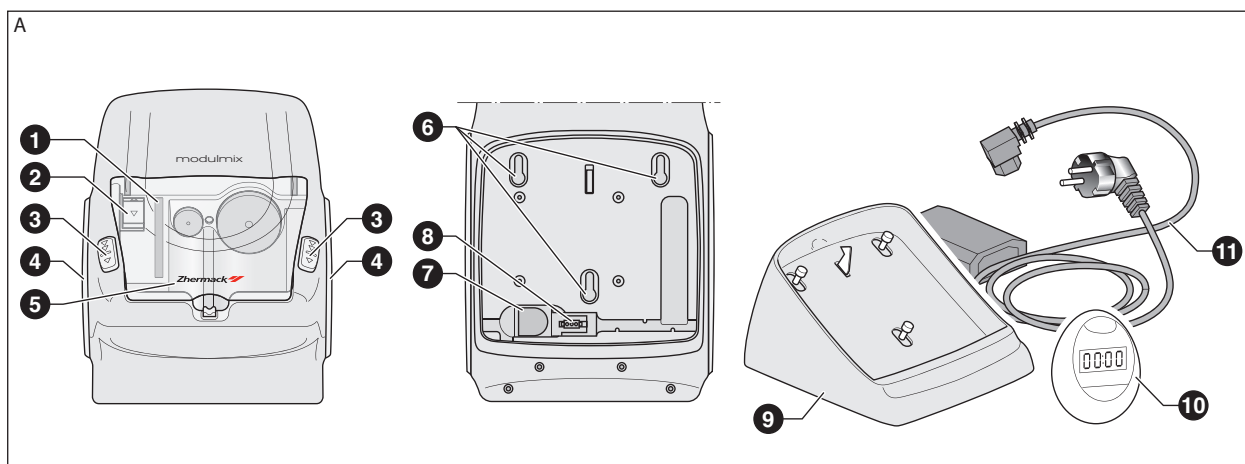
modulmix

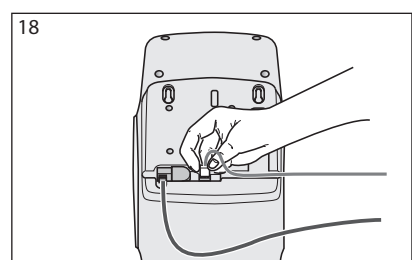
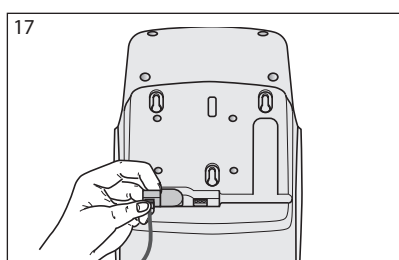
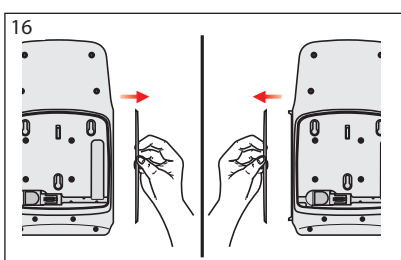
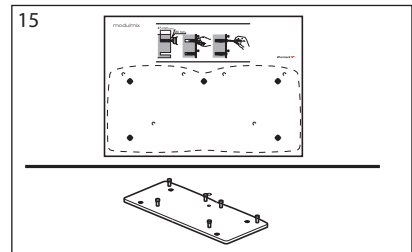
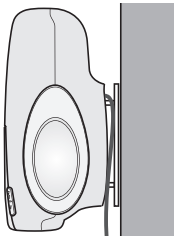
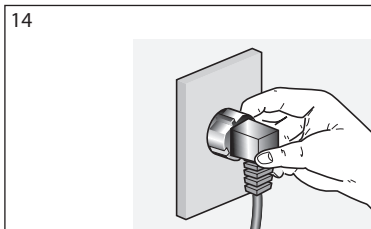
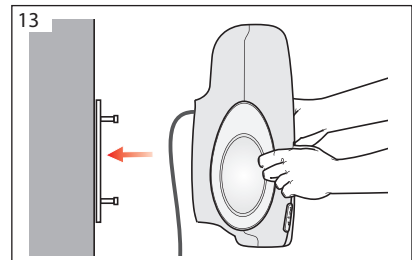
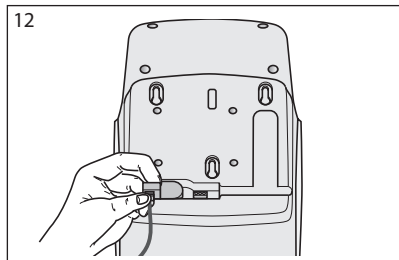
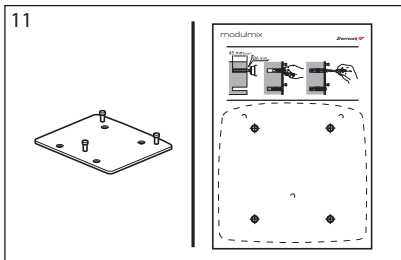
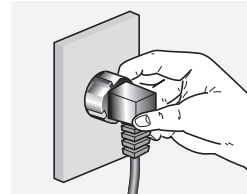
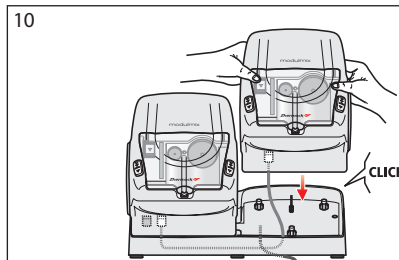
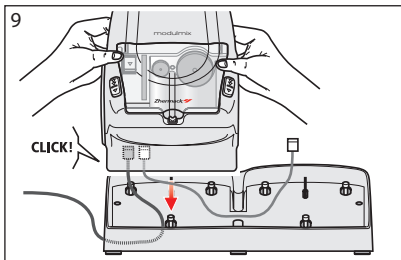
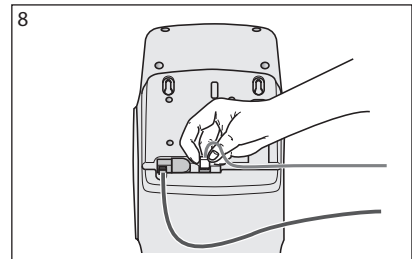
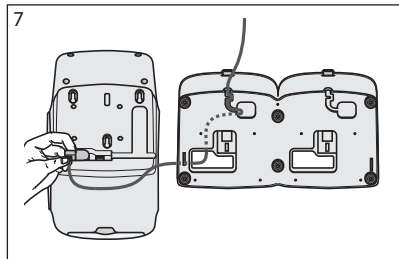
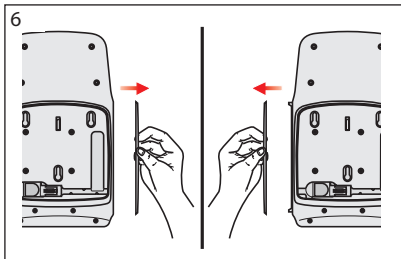
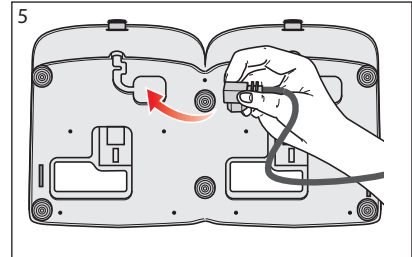
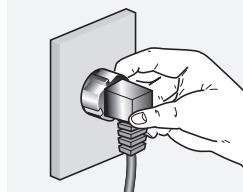
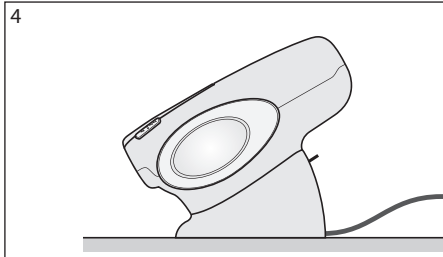
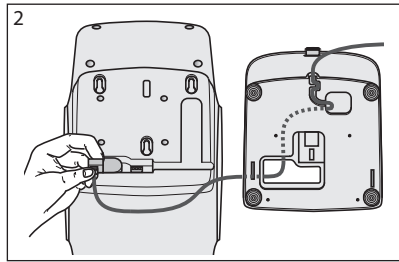
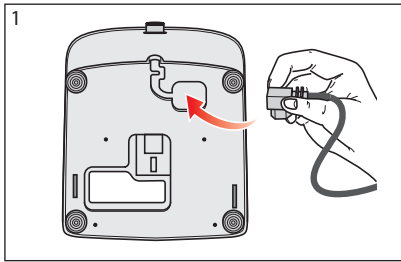
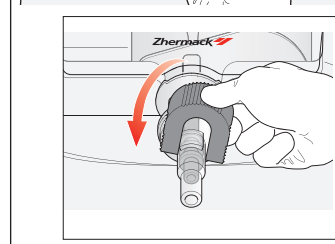
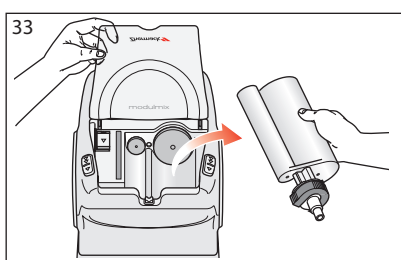
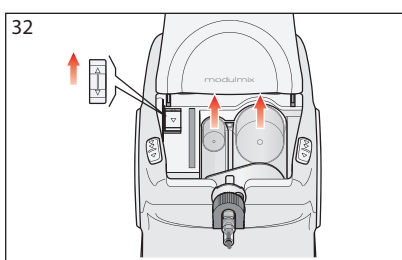
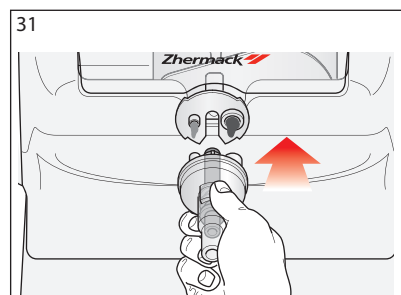
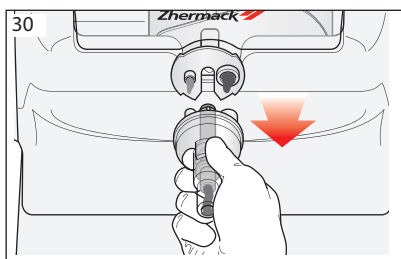
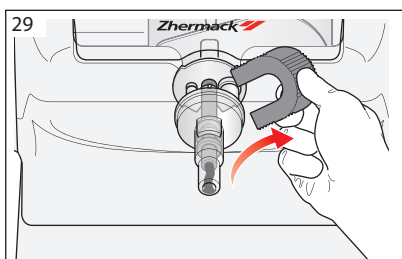
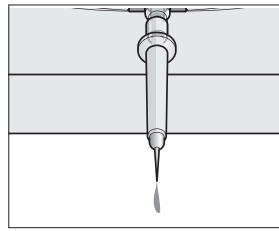
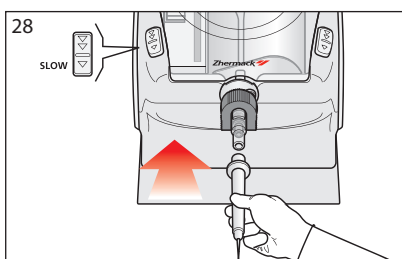
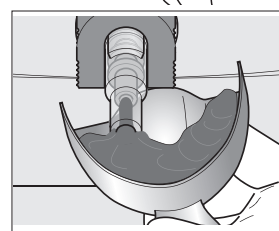
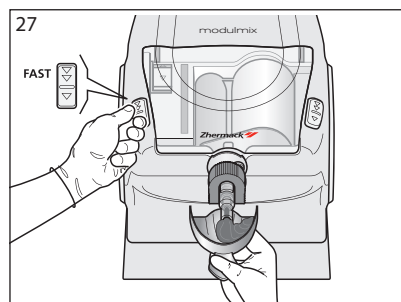
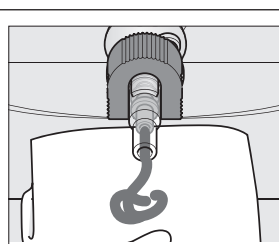
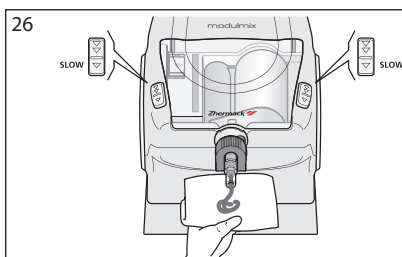
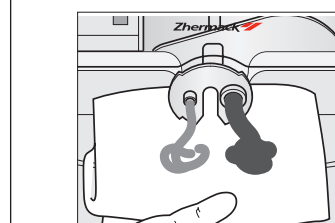
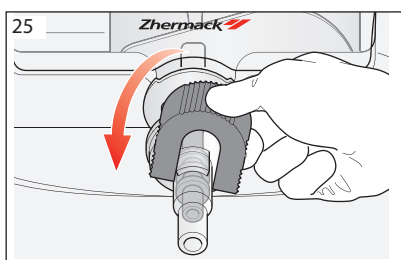
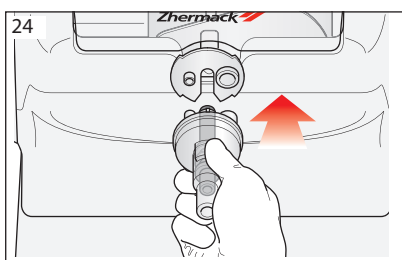
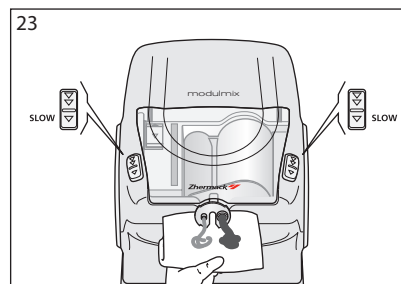
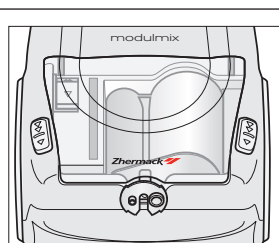
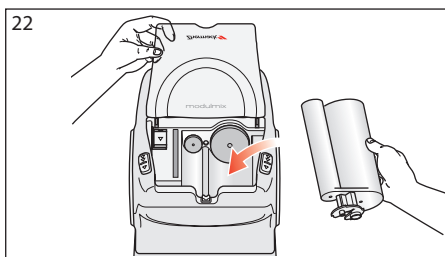
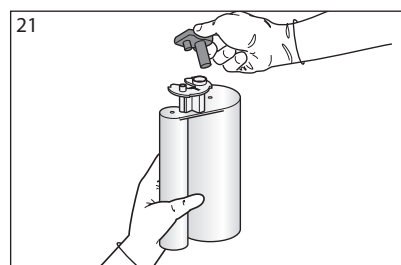
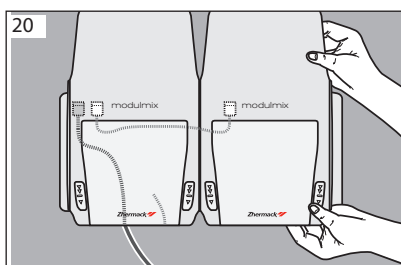
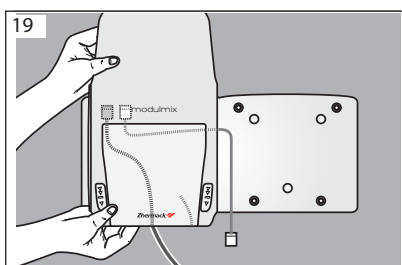
USER MANUAL

Automatic mixing unit

IT	EN	FR	ES	DE	PT	NL	FI	DA	SV	SL	PL
CS	HU	ET	LV	LT	SK	BG	RO	EL	RU	HR	ZH

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IT	7
EN	15
FR	23
ES	31
DE	39
PT	47
NL	55
FI	63
DA	71
SV	79
SL	87
PL	95
CS	103
HU	111
ET	119
LV	127
LT	135
SK	143
BG	151
RO	159
EL	167
RU	175
HR	183
ZH	191

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[illegible]

INDEX

- 1. INTRODUCTION**
 - 1.1 How to read and use the Use and Maintenance Manual
 - 1.2 Terms and symbols used
- 2. GENERAL INFORMATION**
 - 2.1 Liability
 - 2.2 Warranty: general standards
 - 2.3 Authorised staff
 - 2.4 General safety provisions
- 3. DESCRIPTION OF THE MACHINE**
 - 3.1 General description of the machine
 - 3.2 Table and description of the components
 - 3.2.1 Description of the controls
 - 3.3 Identification data
- 4. TECHNICAL FEATURES**
 - 4.1 Technical features and functioning principle
 - 4.2 Technical data
 - 4.2.1 Standard supplies
 - 4.2.2 Accessories available on request
 - 4.2.3 Weight and overall clearance
- 5. TRANSPORT AND HANDLING**
 - 5.1 General warnings
 - 5.2 Packaging and unpacking
 - 5.3 Transport and handling
 - 5.4 Disposal/Dismantling
- 6. INSTALLATION AND COMMISSIONING**
 - 6.1 Warnings and precautions
 - 6.2 Accepted environmental conditions
 - 6.3 Space necessary for use
 - 6.3.1 Worktop
 - 6.4 Power source connection
 - 6.5 Start-up (bench version)
 - 6.5.1 Start-up of modular version (bench)
 - 6.6 Start-up (wall-mounted version)
 - 6.6.1 Start-up of modular version (wall-mounted)
 - 6.7 Declared use/Non-declared and prohibited use
- 7. USING THE MACHINE**
 - 7.1 General warnings
 - 7.2 Safety systems
 - 7.3 Using modulmix
 - 7.3.1 Filling stage
 - 7.3.2 Replacing the mixer tip
 - 7.3.3 Removing the cartridge
 - 7.4 Residual risks
- 8. MAINTENANCE**
 - 8.1 Cleaning the machine
 - 8.2 Routine maintenance
 - 8.3 Extraordinary maintenance
 - 8.4 Re-start after long period of inactivity
 - 8.5 How to request after-sales assistance
- 9. DIAGNOSTICS**
 - 9.1 General warnings
 - 9.2 Troubleshooting
- 10. ATTACHMENTS**
 - 10.1 CE Declaration of Conformity
 - 10.2 Warranty certificate and Identification form
 - 10.3 Technical intervention worksheet

The data given in this Manual can undergo variations without forewarning.

CHAPTER 1: INTRODUCTION

1.1 HOW TO READ AND USE THE USE AND MAINTENANCE MANUAL

EN

This "Use and Maintenance Manual" provides information regarding the installation, use and maintenance of the modulmix mixer, manufactured by Zhermack.

- The machine must be used in compliance with that specified in this Manual: it is therefore recommended that the manual is read carefully prior to installation and start-up. Make sure that you read the entire manual and pay particular attention to messages written in bold type or items highlighted by boxes and/or symbols, which highlight dangerous situations or requiring caution (see paragraph 1.2).
- The respect of standards and recommendations shown in this Manual will ensure safe use of the machine and appropriate interventions on the same.
- The Use and Maintenance Manual is an integral part of the machine: it should therefore be kept together with the machine so that it can be consulted as and when necessary (in a secure dry place away from sunlight, atmospheric agents, etc.) throughout the working life of the machine, even when the machine is sold or finally dismantled.
- It is advised to keep the Manual constantly updated, integrating any amendments, additions or modifications made by the Manufacturer. Any notes or comments should be made on the blank pages at the rear of the Manual (see Attachments: Notes).
- Use the Manual in a way that its content is not damaged.
- Do not remove, tear or write on the Manual for any reason.
- If the Manual is lost or damaged and therefore becomes illegible, request a new Manual from the Manufacturer.

This Use and Maintenance Manual is aimed at operators in order to allow correct use of the product. The Manual contains all useful information for declared use of the machine; in particular the operator will find instructions and indications regarding:

- correct installation of the machine;
- a functional description of the machine and all parts;
- commissioning;
- correct programmed maintenance;
- attention to the most elementary safety and accident-prevention regulations.

This Manual is divided into 9 chapters, each referring to a specific subject inherent to the entire machine.

The attachments are found at the rear of the manual. They complete the documentation.

1.2 TERMS AND SYMBOLS USED

It is fundamental that maximum attention is paid to the following symbols and their meaning because they highlight important information such as, critical situations, practical advice or straightforward information. Consult this page if in doubt regarding the meaning of the symbol when reading the Manual.

- Any operations carried out without compliance to instructions or tampering with machine parts, which can cause serious injury to persons, will be highlighted with:

DANGER!

This symbol refers to Safety Standards that **MUST** be followed in order to guarantee your safety and that of others and to prevent damage to the machine.

- Critical situations concerning dangerous areas, situations or operations in which failure to comply with the instructions may render the warranty null and void, cause damage to machine, connected components or the surrounding area, are highlighted by the symbol:

ATTENTION!

This symbol refers to Safety Standards that must be followed carefully in order to guarantee your safety and that of others and to prevent damage to the machine.

- Any operations carried out without compliance to instructions or tampering with machine parts, which can cause damage to the machine rendering the warranty null and void, will be highlighted with:

PROHIBITED!

This symbol calls attention to operations that are prohibited.

- Any general information and/or useful advice referring to any subject, will be highlighted with:

NOTE!

This symbol highlights information and/or useful advice.

CHAPTER 2: GENERAL INFORMATION

2.1 LIABILITY

Failure to comply with the instructions contained in this Use and Maintenance Manual relieves the Manufacturer from any liability. For any information not included or that cannot be deduced from the Manual please contact the Manufacturer directly.

Zhermack S.p.A.

Via Bovazecchino, 100

45021 Badia Polesine - RO - Italy

Tel. +39 0425 597 611, Fax +39 0425 53 596

<http://www.zhermack.com>, e-mail: info@zhermack.com

In particular, if machine maintenance is carried out in a way that does not comply with the instructions provided or in a way to jeopardise integrity or modify the features, Zhermack is relieved from any liability inherent to safety of persons and faulty operation of the machines.

2.2 WARRANTY: GENERAL STANDARDS

Find the Warranty Certificate and the Identification Form attached to this Use and Maintenance Manual, which must be filled in completely.

Generally the warranty is rendered null and void by the following:

- improper use of the machine;
- incorrect installation;
- negligence of envisioned maintenance;
- modifications or interventions not authorised by the Manufacturer (especially on safety devices);
- use of non-original spare parts.

IMPORTANT: Always keep the original product packaging throughout the guarantee period. You will need the packaging if the product has to be returned to the Service Centre for repairs or replacement in the event of manufacturing defects or malfunction (with the exception of wilful damage, negligence, failure to perform routine maintenance or damage during transport). If you are not in possession of the original packaging, contact Zhermack and ask for instructions (possible repackaging costs will be charged on delivery). The customer is responsible for all costs associated with unsuitable packaging of the product to be returned to the manufacturer. Zhermack is not liable for faults, malfunctions or damage to the product deriving from or consequent to the use of unsuitable packaging, or packaging not of a quality equal to or better than the packaging used for the original shipment.

As of this moment Zhermack is relieved from all liability if any type of modification or tampering is made on the machine without previous written authorisation from the Manufacturer. Consequently, any repairs made by unauthorised staff, use of non-original spare parts or failure to comply with the installation standards specified in this manual shall immediately render the warranty null or void.

2.3 AUTHORISED STAFF

The staff in charge of the machine is divided into the following categories:

Operator

Person trained to use the machine. He performs all tasks necessary for operation, such as: running/activation of start/stop controls, cleaning, inspection and simple interventions linked to normal machine use.

Before using the machine or carrying out any intervention on it, the operator must read the entire manual carefully. This manual has been drawn up to provide the user with essential information on rational, safe use of the machine. The user must comply with the information given at all times. The user must also use the worksheets, attached to this manual, to keep a record of maintenance interventions, component replacement as well as make notes regarding anomalous functioning.

Maintenance technician authorised by Zhermack

Person who operates in all working conditions and at all safety levels to perform repairs/ mechanical and electric adjustments, periodic preventive maintenance and replacement of any components.

2.4 GENERAL SAFETY PROVISIONS

The operator must comply with the advice and warnings given below in order to ensure safe, long-lasting machine performance:

- do not allow untrained staff to use the machine;
- prepare an optimal worktop, making sure that the operational space is free from obstacles, clean and has suitable lighting;
- avoid distractions when using the machine;
- in the event of a malfunction follow the advice given in this manual; if in doubt, stop the machine and contact your Authorised After-Sales Centre immediately;
- never use a faulty machine;
- do not open machine compartments containing electric parts;
- do not replace the power supply cable or tamper with the supplied plug;
- it is prohibited to work objects with features different from those for which the machine has been designed;
- before carrying out any cleaning, routine and/or extraordinary maintenance, make sure that the machine has been unplugged from the mains power supply;
- always make sure that there are no foreign bodies inside the work area. These could prevent damage to the machine and/or injury to the staff;
- do not use petrol or flammable solvents as detergents; use only non-flammable, non-corrosive and non-toxic substances. (see paragraph 8.1).

- Zhermack cannot be held responsible for any damage to the equipment owing to incorrect maintenance performed by unskilled staff or maintenance that does not comply with that stated in this manual.
- As of this moment Zhermack is relieved from any injury/damage to persons/objects that should occur following any different use and use not declared in this manual.

The machine has an electrical plant therefore, in the event of a fire, no matter how small, only use powder extinguishers. Do not use water.

CHAPTER 3: DESCRIPTION OF THE MACHINE

3.1 GENERAL DESCRIPTION OF THE MACHINE

The machine described in this Manual is the modulmix mixer, designed to automatically batch and mix dental impression silicones, which are packaged in cartridges with a mixing ratio of 5:1.

3.2 TABLE AND DESCRIPTION OF THE COMPONENTS - Fig. A

1. Cartridge level indicator
2. Fast pistons advance/retraction control
3. Slow/fast extrusion control
4. Coloured side covers (can be customised)
5. Door with safety micro switch
6. Connections for base (bench version) or plate (wall-mounted version)
7. Main socket for connection to the main power supply
8. Secondary socket for modular connection
9. Bench-type support base
10. Digital timer
11. Power supply

3.2.1 DESCRIPTION OF THE CONTROLS - Fig. B

1. Rapid piston retraction
2. Rapid piston advance
3. Fast extrusion
4. Slow extrusion

3.3 IDENTIFICATION DATA

Fig. C: The machine described in this manual has a plate giving identification data of the same and the Manufacturer:

- A Name and address of the Manufacturer
- B Machine code and model
- C Serial number
- D Power supply information

The machine has been built in compliance with the EEC Directives stated in the attached "CE Declaration of Conformity. Use the identification data to request spare parts, information and After-Sales Assistance from authorised centres. If the plate should deteriorate with use and become illegible, even partially, request a replacement from the Manufacturer stating all data.

Do not remove and/or ruin the plate so that the machine identification data can be recognised at all times.

CHAPTER 4: TECHNICAL FEATURES

4.1 TECHNICAL FEATURES AND FUNCTIONING PRINCIPLE

Zhermack creates qualitatively and technologically advanced work instruments, in away that the user feels that they are made-to-measure his everyday working requirements. Zhermack uses state-of-the-art instruments in the design field when making these products, thus ensuring maximum machine functionality. The use of top quality materials and performance of all inspections necessary for operator safety guarantee Zhermack products, making them competitive at a worldwide level. Modulmix has a cutting-edge compact and linear design which guarantees reduced encumbrance upon work surfaces; the transparent plastic door allows immediate inspection of the material being used and the quantity of material contained in the cartridge. Moreover, the double control buttons for the 2-speed motor, positioned on the right and left, near the extrusion zone, allow visual checks during filling of the impression tray and make work stage easier. Finally, modulmix has a button for motorised piston advance or retraction in order to speed up the cartridge change process.

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Power voltage	230 V ~ ± 10% 50/60Hz (or 115 V ~ 50/60 Hz)
Absorbed power	160 Watt
Protection rating IP	IPX0
Noise level	<70 dB
Overvoltage category	II
Applicable pollution degree	2

4.2.1 STANDARD SUPPLIES

Components	Q.ty	Ord. Code*
230V~ power cable	1	XR0240630
115V~ power cable (US version only)	1	XR0240615
Bench-type support base	1	XR0240505
Digital timer	1	XR0240580

(*) Order Code to be used when ordering spare parts.

4.2.2 ACCESSORIES AVAILABLE ON REQUEST

Components	Q.ty	Ord. Code*
Light blue side covers	2	XR0240550
Orange side covers	2	XR0240551
Blue side covers	2	XR0240552
Green side covers	2	XR0240553
White side covers	2	XR0240554
Wall-mounting kit (with drilling template)	1	XR0240520
Modularity kit (hook and connection cable)	1	XR0240510
Bench support for two modulmix machines	1	XR0240500
Wall-mounting kit (with drilling template - modular version -	1	XR0240530

(*) Order Code to be used when ordering spare parts.

4.2.3 WEIGHT AND OVERALL CLEARANCE

Overall clearance without packaging	
Height	mm. 350
Width	mm. 230
Depth	mm. 370
Empty weight	Kg. 12,5

CHAPTER 5: TRANSPORT AND HANDLING

5.1 GENERAL WARNINGS

In order to prevent injury/damage to persons/objects, pay great care when handling the machine, scrupulously complying with all precautions and instructions contained in the following paragraphs.

5.2 PACKAGING AND UNPACKING

Fig. D: All material is carefully controlled by the Manufacturer before delivery. On receiving the machine, make sure that it has not been damaged during transport and that the packaging has not been tampered with, resulting in missing parts. Should any damage or missing parts be noted, inform the Dealer and the Carrier immediately by forwarding photographic evidence. The packaging, made up of a cardboard box with expanded plastic internal protections, contains:

- the modulmix mixer;
- the material supplied (see par. 4.2.1);
- the Use and Maintenance Manual;
- the quick guide.

IMPORTANT: Always keep the original product packaging throughout the guarantee period. You will need the packaging if the product has to be returned to the Service Centre for repairs or replacement in the event of manufacturing defects or malfunction (with the exception of wilful damage, negligence, failure to perform routine maintenance or damage during transport). If you are not in possession of the original packaging, contact Zhermack and ask for instructions (possible repackaging costs will be charged on delivery). The customer is responsible for all costs associated with unsuitable packaging of the product to be returned to the manufacturer. Zhermack is not liable for faults, malfunctions or damage to the product deriving from or consequent to the use of unsuitable packaging, or packaging not of a quality equal to or better than the packaging used for the original shipment.

5.3. TRANSPORT AND HANDLING

If the machine must be moved:

1. disconnect the machine from the electric plant;
2. hold the machine (see par. 6.5 Fig. 1), always keeping it in a vertical position. Regarding this, pay attention to the weight and clearance of the machine (see par. 4.2.3), in order to prevent any type of damage occurring due to the failure of considering these values;
3. pack the machine in the original packaging. If this is not possible, take all necessary precautions to protect the machine from knocks and drops, as the goods are transported at the owner's risk.

- Failure to comply with the rules stated above relieve the Manufacturer from any liability regarding machine malfunctioning and consequently makes the warranty null and void.
- When shipping and/or delivering the machine to an Authorised After-Sales Centre always enclose a copy of the purchase document and a copy of the properly filled-in identification form.
- If the machine is transported by carrier, the Postal Service or rail, it is recommended to insure the shipment.

5.4 DISPOSAL/DISMANTLING

When disposing of the packaging material the user must follow the regulations in force in the country of use relative to the following materials:

- wood/paper: non-polluting materials, but must be re-cycled correctly;
- polystyrene/plastic: pollutant material, do not burn (toxic fumes) or dispose of in the environment. Dispose of these materials in compliance with the regulations in force in the country of use.

If the machine is to be dismantled, the operator must be in charge of dismantling, elimination and re-cycling of the following materials in compliance with the EEC Directives and the laws in force in the country of use:

- plastic parts; steel parts; glass parts; insulated electric cables; rubber parts;
- there are no toxic or corrosive substances.

The product must not be disposed of as household waste, but at the end of its life must be collected separately in compliance with the 2012/19/UE Directive.

CHAPTER 6: INSTALLATION AND COMMISSIONING

6.1 WARNINGS AND PRECAUTIONS

Before carrying out installation, make sure that all safety conditions exist and follow the indications given below carefully.

For greater clarity, the numbers stated in the figures positioned near to the paragraph boxes correspond to the numbering of the data in Chapter 3, regarding the "Description of the machine".

6.2 ACCEPTED ENVIRONMENTAL CONDITIONS

Unless stated otherwise, at the time of ordering, it is intended that the machine must be envisioned to function regularly in the following environmental conditions:

Place of Use Indoors
Altitude Up to 2000 m
Working temperature From 5°C to 40°C
Humidity Max 80%

Conditions different to those prescribed can cause anomalies or sudden breakdown. Lighting in the installation area must be sufficient to guarantee

good visibility in all points of the machine. Luminosity not less than 200 lux must be guaranteed. The lighting must be uniform without reflections, which could dazzle the operator.

The machine is not prepared and/or designed to operate in environments with explosive atmosphere or with the risk of fire. However, if a fire should accidentally occur, act as described in paragraph 2.4.

The working area lighting plant is an important factor for the safety of persons and the quality of work. In Italy this subject is governed by a Ministerial Decree that clearly determines the average level of prescribed lighting. In other countries the regulations in force are valid regarding accident-prevention and hygiene in the workplace.

6.3 SPACE NECESSARY FOR USE

The choice of the place and space required for installation is an important factor for the quality of work and safety and for correct maintenance activities.

This area must not only be spacious enough to allow for optimum machine operation - also during cleaning and maintenance phases - but must also be well lit, aired, not dusty and not exposed to direct sunlight. Note also that the unit must be positioned so the connection plug can be handled/manipulated with ease.

6.3.1 WORKTOP

Fig. E - BENCH VERSION

Modulmix has been designed to work on a worktop parallel to the ground. Suitable worktops include service furniture (operator is standing) or modelling tables (operator is sitting). All worktops must be checked for stability.

Fig. F - WALL-MOUNTED VERSION

Modulmix is also designed to be wall-mounted. For fast, accurate drilling of the holes, use the supplied drilling template; check the horizontal alignment of the holes. Once drilling has been concluded, introduce the plugs in the holes and then screw in the screws, leaving them to project from the wall by 8-10 mm. Check machine stability before use.

- When choosing wall plugs, bear in mind the weight of the machine as stated in section 4.2.3 as well as the characteristics of the wall on which it is to be installed.
- Use specific plugs if mounting the machine on plasterboard walls.

6.4 POWER SOURCE CONNECTION

The connection is made using the power supply cable. Check:

- that mains power supply voltage and frequency correspond to those indicated on the identification plate (an incorrect power supply voltage may damage the machine);
- that the power supply mains has a suitable earth connection.

Do not tamper with the power supply cable and relative plug. For replacement of the same due to damage and/or wear, only contact Authorised After-Sales Centres.

An earth connection is mandatory. If an extension must be used temporarily, it must be in compliance with the standards in force in the country of use.

6.5 START-UP (BENCH VERSION)

Fig. 1: Insert the cable into the appropriate cavity in the base.

Fig. 2: Connect the power cable to the socket under the machine.

Fig. 3: Fix the modulmix to the base.

Fig. 4: Connect the power cable to the socket.

6.5.1 START-UP MODULAR VERSION (BENCH)

Fig. 5: Insert the cable into the appropriate cavity in the double base.

Fig. 6: Remove the internal side covers.

Fig. 7: Connect the power cable to the socket under the machine.

Fig. 8: Engage the connection cable with the first modulmix.

Fig. 9: Fix the first modulmix to the double base.

Fig. 10: Connect the cable to the second modulmix and fix it to the base.

6.6 START-UP (WALL-MOUNTED VERSION)

Fig. 11: Use the template and the bracket for wall-mounting.

Fig. 12: Connect the power cable to the socket under the machine.

Fig. 13: Fix the machine to the bracket using the relevant attachments.

Fig. 14: Connect the power cable to the socket.

6.6.1 START-UP MODULAR VERSION (WALL-MOUNTED)

Fig. 15: Use the template and the modular bracket to drill the wall correctly.
 Fig. 16: Remove the internal side covers.
 Fig. 17: Connect the power cable to the socket under the machine.
 Fig. 18: Engage the connection cable with the first modulmix.
 Fig. 19: Fix the first modulmix to the modular plate.
 Fig. 20: Connect the cable to the second modulmix and fix it to the plate.

6.7 DECLARED USE, NON-DECLARED AND PROHIBITED USE

The modulmix mixer has been designed to:

- automatically batch and mix dental impression silicones, which are packaged in cartridges with a mixing ration of 5:1.

Using the machine for any purpose other than that described in this manual shall be considered improper and thus forbidden. The improper use of the machine renders the warranty null and void and Zhermack declines all liability for any damage to objects and injury to operators or third parties.

The main reasons for which the warranty is rendered null and void are given in paragraph 2.2 and in the "Warranty Certificate" attached to this Manual.

CHAPTER 7: USING THE MACHINE

7.1 GENERAL WARNINGS

Never use the machine if parts have been disassembled: before use always check the perfect positioning of each individual machine component.

- For total certainty of safe operation, scrupulously follow the work phases given below.
- For the extrusion of silicones packaged in aluminium cartridges always use the slow mode.

The high performance of modulmix is guaranteed using Zhermack silicones.

7.2 SAFETY SYSTEMS

With the purpose of ensuring maximum reliability, Zhermack uses only the best materials and components to build its machines, putting them through stringent tests prior to delivery. All materials are top quality and their introduction into the company, storage and use in the workshop is controlled constantly in order to guarantee there is no damage, deterioration and/or malfunctioning.

All construction elements, connections and controls have been designed and realised to a high safety levels that can resist anomalous stress or however higher than those indicated in this manual.

It is prohibited to use the machine for any other use than that specified in this manual. The Manufacturer is exonerated from any liability for damage of any nature caused by the incorrect and/or different use of the machine from that stated above.

7.3 USING MODULMIX

Fig. 21: Remove the clip cap from the cartridge.
 Fig. 22: Open the door (making sure the pistons are retracted) and insert the cartridge inside. Advance the pistons and close the door.
 Fig. 23: Press the "slow" key and extrude the product until homogeneity between "BASE" and "CATALYST" is obtained.
 Fig. 24: Insert and fix the dynamic-static mixing tip onto the mixing shaft correctly.
 Fig. 25: Insert the locker for fixing of the dynamic-static mixing tip.
 Fig. 26: Extrude a small amount of material (in "slow" mode) before filling the impression tray.

7.3.1 FILLING STAGE

Fig. 27: Fill the impression tray making sure that the point of the dynamic-static mixing tip is left submerged in the silicone.
 Fig. 28: To use single-phase material fill the supplied syringe and insert it into the dynamic-static mixer tip at "slow" speed.

7.3.2 REPLACING THE MIXING TIP

Fig. 29: Turn the locker 90° and remove it.
 Fig. 30: Remove the disposable dynamic-static mixing tip.
 Fig. 31: Insert a new mixing tip correctly and fix it using the locker.

7.3.3 REMOVING THE CARTRIDGE

Fig. 32: Open the door and activate retraction in order to withdraw the pistons from inside the cartridge.
 Fig. 33: Remove the cartridge.

To insert and remove the cartridge easily, keep the fast pistons advance/retraction control button fully pressed.

7.4 RESIDUAL RISKS

Residual risks are:

- a danger that cannot be totally eliminated through design and the application of safety devices;
- a potential danger that cannot be highlighted.

In order to prevent accidents, always operate with respect to that described in this Manual. If in doubt, always contact the Manufacturer or Authorised After-Sales Centre.

CHAPTER 8: MAINTENANCE

8.1 CLEANING THE MACHINE

Use a dry cloth to clean the machine exterior. If necessary, dampen the cloth with a small amount of water or a non-degreasing detergent (see paragraph 2.4). The operator must ensure that the machine is kept clean and free from any foreign bodies such as debris, water, non-conducting liquids etc. For this purpose cleaning must be performed at the end of every shift, with the machine in stable conditions.

- Before cleaning the outside of the machine, always make sure that the master switch is off. It is prohibited to clean the machine when it is connected to the electric power supply mains.
- Do not use inflammable, corrosive, alkaline or toxic liquids to clean the machine.

8.2 ROUTINE MAINTENANCE

Once the machine has been cleaned (see paragraph 8.1), the operator must check for worn, damaged or loose parts. If any such situations are detected, contact the Authorised After-Sales Centre.

- If any of the previously-described situations should occur, it is prohibited to re-start the machine before the problems have been solved.

In particular, before moving away from the machine, the operator detecting problems of any nature must place a sign on the it indicating that maintenance is in progress and that use is prohibited (EC-compliant signs can be purchased from the relevant suppliers).

Routine maintenance, cleaning and correct machine use are essential factors in guaranteeing long-lasting machine performance and safety.

- Machine safety devices/guards must never be removed except for repairs and/or extraordinary maintenance. They must be restored as soon as the reasons for their temporary removal no longer exist and in all cases before the machine is used again.

8.3 EXTRAORDINARY MAINTENANCE

Extraordinary maintenance includes the repair of any accidental breakage and the replacement of worn or malfunctioning parts.

All electrical and mechanical interventions must only be performed by a Zhermack authorised maintenance technician.

Any modification that alters machine features from a safety and risk-prevention point of view can only be carried out by the Manufacturer who will then certify conformity of the machine with safety standards. Therefore any modification or maintenance intervention not considered in this manual is to be considered prohibited. Any modification, tampering or repair performed by unauthorised staff along with the use of non-original spare parts relieve the Manufacturer from any liability. If the necessary maintenance interventions are not considered in this Use and Maintenance Manual, contact the Authorised After-Sales Centre.

8.4 RE-START AFTER LONG PERIOD OF INACTIVITY

If the machine is re-started after a particularly long period of inactivity, operate as indicated in Chapter 6 regarding commissioning. If the machine functions anomalously, act as described in Chapter 9 regarding "Diagnostics": if the problem should persist or is not considered, contact the Authorised After-Sales Centre immediately.

8.5 HOW TO REQUEST TECHNICAL AFTER-SALES ASSISTANCE

Choose one of the following options to request after-sales assistance:

1. contact the Authorised After-Sales Centre;
2. telephone the Zhermack After-Sales Service at +39 0425.597.611 and proceed as follows: give personal data (e.g. address and telephone number); state the information given on the machine identification plate (see paragraph 3.5); explain the problem with the machine clearly;
3. fax all of the information stated in the previous point and the correctly filled-in identification form (see Attachment 10.2) to the following n° +39 0425.53.596.

CHAPTER 9: DIAGNOSTICS

9.1 GENERAL WARNINGS

If any of the anomalies described below should occur, solve the problem, where allowed, with respect to all instructions contained in this manual. If the problem persists, contact the Authorised After-Sales Centre.

9.2 TROUBLESHOOTING

FAULT **No material extrudes from the mixing tip when the activation button is pressed.**

CAUSE

1. Part of the material has hardened in the cartridge.
2. The material has hardened in the mixing tip (also previously used).
3. The pistons are blocked or incorrectly positioned.
4. There is no more material inside the cartridge.

REMEDY

1. Replace the cartridge.
2. Use a new mixing tip.
3. Press the piston activation button repeatedly to position them correctly.
4. Check the cartridge level indicator. Replace the cartridge if necessary.

FAULT **The cartridge cannot be removed or introduced.**

CAUSE

1. The pistons have not returned to the start position.
2. The pistons are blocked inside the cartridge and motorised retraction does not work.

REMEDY

1. Press the activation button and make the pistons go back to the start position until they stop automatically.
2. Remove the plug from the mains. Remove the right side cover and use an Allen wrench to manually release motorised retraction.

FAULT **The mixing tip does not turn.**

CAUSE

1. Hardened material in the mixing tip.
2. The mixing tip is not correctly inserted into the cartridge.

REMEDY

1. Use a new mixing tip.
2. Check correct positioning between the mixing tip and the mixing shaft.

FAULT **The machine does not work.**

CAUSE

1. Power supply cable malfunctioning.
2. The door is open or not closed properly.

REMEDY

1. Contact the Authorised After-Sales Centre.
2. Close the door properly.

CHAPTER 10: ATTACHMENTS

10.1 EC DECLARATION OF CONFORMITY

The Manufacturer:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597611 • Fax +39 0425 53596

Person authorised to compile the technical file:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597611 • Fax +39 0425 53596

declares under its sole responsibility that the product:

modulmix

is in conformity with Directive 2011/65 EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

It is also in conformity with the laws implementing the following directives:

Medical Devices:
Directive 93/42/EEC - Class I

Please note that the manufacturer's liability for damages caused by a defective product expires 10 years from the date on which the manufacturer released such product on the market and that such liability is transferred to the user in accordance with Directive 85/374/EEC, subsequent amendments and the laws implementing it.

Badia Polesine, 25/02/14

Paolo Ambrosini
General Manager

10.2 WARRANTY CERTIFICATE AND IDENTIFICATION FORM

1. With this document the Manufacturer certifies that the product has been built correctly, top quality materials have been used and all inspections necessary for compliance with European Community Directives have been performed. The product is covered by a warranty period of 24 months from the date of delivery to the user. Proof of this date is provided from the purchase document and the copy of the identification form. These documents must be shipped to the After-Sales Centre with the machine. Persons wishing to make use of the warranty must inform of the fault within 8 days of its discovery for the effects and purpose of art. 1495 of the Civil Code. The warranty is limited to replacement or repair of individual components or pieces with manufacturing defects and does not cover costs related to transfer of technical staff, transport, packaging etc. Breakdowns, faults or damage deriving from poor maintenance, incorrect power supply, negligence, inexperience and other causes not imputable to the Manufacturer and parts subject to normal wear are not covered by the warranty. The warranty does not cover breakdowns caused by failure to carry out routine maintenance due to customer negligence. This warranty does not provide for any reimbursement for direct or indirect damages of any kind caused to persons or objects due to machine inefficiency.

2. The warranty is automatically rendered null and void whenever the machine is repaired, modified or tampered with by the purchaser or unauthorised third parties. The liability for damage caused by faulty products expires after 10 years from the date in which the Manufacturer put the products on the market, after this time it becomes the user's responsibility, as envisioned by the 85/374/EEC Directive.

3. To have work carried out under warranty the purchaser must only contact the dealer, Manufacturer-authorised After-Sales Centres or the Manufacturer itself.

The warranty gives the right for free replacement of faulty parts. It does not, however, give the right for replacement of the entire machine.

4. In the case of disputes regarding application of the warranty, quality or conditions of the machine delivered, the purchaser cannot suspend or delay total payment or payment of the instalments.

5. No reimbursement can be requested by the purchaser for loss of production.

6. The warranty is rendered null and void if:

- The machine is damaged by a fall, exposure to fire, spilt liquids lightening, natural disasters or, however, causes not imputable to manufacturing defects.
- Incorrect installation.
- Incorrect mains connection (incorrect power supply nominal voltage) or suitable protection devices have not been installed.
- The serial number has been removed, cancelled or altered.

7. The components to be replaced under warranty must be returned to Zhermack which will then send the appropriate spare parts. Any changed parts that are not returned will be charged.

8. The Manufacturer and the Dental Deposit do not have to supply replacement machines during the repair period.

9. For tax purposes spare parts will only be provided under warranty where the relevant limitations set out in the terms and conditions of the warranty are respected.

10. For any other eventuality not provided for by this Warranty Certificate and the regulations, consult the Italian Civil Code.

11. Invoices for labour, travel and call-out must be paid on their receipt.

12. The manufacturer and the dealer will comply with all data processing laws in force, including those relative to safety measures and in compliance with the specifications given in the data processing information sheet.

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