**Zhermack**   
Dental

# VILLACRYL S

---

self-curing acrylic resin  
for denture repairs

Last Update-2018-03

**Zhermapol Sp. z o.o.**

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T +48 22 858 82 72 | [biuro@zhermack.com](mailto:biuro@zhermack.com)

[www.zhermapol.pl](http://www.zhermapol.pl)

 0120

EN | FR | ES | DE | PT | NL | FI | EL  
DA | SV | SL | RU | PL | CS | TR | RO  
HR | HU | SK | LV | LT | ET | AR | BG | UK

**Self-curing acrylic resin for denture repairs**

**Product characteristics:** Seat-curing acrylic material for repairs of teeth dentures.

**Classification accordance EN ISO 20795-1:** Type 2, Class 1

**Designation:** Repair of complete dentures, acrylic parts in frame dentures and acrylic removable orthodontic appliances.

**Contraindications:** In people sensitive to the ingredients of the product allergic reaction may occur. If you notice any symptoms of allergic reaction should stop using the product and consult a doctor. Do not repair the relining and other activities using direct method in the patient's mouth.

**Side effects:** Not found in current clinical use of the preparation.

**Colour shades**

0 - transparent

V2 - milk pink veined

V3 - dark pink veined

V4 - pink veined

T4 - pink

**Assortment\***

REF	Set
V1300Z01	Villacryl S colour 0 (100 g powder + 50 ml liquid)
V1300V2Z04	Villacryl S colour V2 (100 g powder + 50 ml liquid)
V1300V3Z08	Villacryl S colour V3 (100 g powder + 50 ml liquid)
V1300V4Z05	Villacryl S colour V4 (100 g powder + 50 ml liquid)
V1300T4Z03	Villacryl S colour T4 (100 g powder + 50 ml liquid)
	<b>Powder</b>
V130V4P06	Villacryl S colour V4 powder 1 kg
	<b>Liquid</b>
V130L02	Villacryl S liquid 200 ml
V130L07	Villacryl S liquid 500 ml

\* Information about the availability of assortment is provided by distributor.

**Chemical composition**

**Powder:** acrylic polymers, benzoyl peroxide, pigments.

**Liquid:** methyl methacrylate, ethylene glycol dimethacrylate, N,N-dimethyl-p-toludyne.

**Method of use**

**Preparation:** Damaged place of the denture to be cleaned and damped with monomer.

**Recommended following mixing proportions:** 10 g of power and 5 g (5,3 ml) of liquid.

**Mixing:** Use a sealable bowl made of chemical-resistant plastic, ceramic, porcelain, glass or stainless steel to mix the components. Pour the liquid into the bowl first and the powder second. Mix properly with spatula, than leave it in closed container.

**Making:** Use a vessel made of chemical-resistant plastic, ceramic, porcelain, glass or stainless steel to mix the components. Pour the liquid into the vessel first and the powder second. Powder and liquid to be mixed precisely using spatula and leave it in closed vessel for 1 - 2 minutes in temperature 20 - 23°C.

**Laying on:** For hands protection it is recommended to use gloves from polyethylene HDPE.

From time to time check the suitability of the material to be laid on in repair place, especially when the ambient temperature is higher than 20 - 23°C. In case if dough fast drying (high ambient temperature) damp with monomer the surface of dough placed on the denture.

**Polymerization:** Material curing in room temperature. The autopolymerization time is around 18 minutes. Better mechanical properties of the material can be achieved using during polymerization process pressure vessel for repairs or pressure polymerizator. In such case the material must be polymerized in wafer temperature 50 - 60°C with 2 bars pressure for 15 minutes. Residual methyl methacrylate monomer: < 2,7%

The residual monomer content could be reduced by using the pressure polymerization.

**Treatment:** Use standard treatment method. Grind with pumice and polish with polishing paste from Zhermapol.

**Important:**

- Accurately prepare repaired denture place with standard method.
- Mixed powder with liquid, keep in closed vessel.
- Time of preparing dough depends on ambient temperature.
- The repaired place wet with monomer to ensure proper bonding.

**Danger and counteraction**

**Information for dental technicians:**

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P210 Keep away from heat/sparks/open flames/hot surfaces. - No smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P302+P352 IF ON SKIN: The area of product contact with skin wash thoroughly with soap with water.

P501 Dispose of contents/container to an approved waste disposal plant.

While working with the material do not: consume meals or smoke tobacco. Work should be carried out in good ventilated premises. Use standard ventilation. In case of contact of the powder particles with eyes mechanical irritation may occur. In case of contact acrylic powder with eyes wash it with water and contact physician. Liquid contains methyl methacrylate - flash point +10°C. Avoid direct contact of the acrylic dough on the skin. The area of wet acrylic dough contact with skin wash thoroughly with soap water. In case of accidental swallowing contact the physician. Do not pour liquid into drainage system.

**Information for dentists:** Possibility to pass on infection. Each denture must be cleaned and disinfected before placing in the

oral cavity. It is important to inform the user that it is necessary to put prosthesis into water, when it is not used. Especially during first two weeks. It is important to inform about hygiene connected with use of prosthesis. For denture hygiene may be used special denture cleaning brushes, odorless soap, various preparations for cleaning dentures and ultrasonic cleaners.

**Storage and handling:** Liquid is sensitive to influence of temperature and UV radiation under the influence of which the uncontrolled reaction of polymerization may occur. Product components should be stored in original containers, in dark and good ventilated compartment with temp. 5-25°C, in an area not accessible by children. Do not use the product after expiry date specified on the sales packaging and labels of the product ingredients.

**Disposal considerations:** Dispose of in accordance with applicable regulations. Carton packaging, labels, instructions - the possibility of recycling, dispose of as communal not dangerous waste. Powder component and a packaging are not dangerous. Dispose of as communal not dangerous waste. In case of plastic container - the possibility of recycling. Liquid and liquid packaging - hazardous waste. Pass to an authorized waste contractor.

**Note! For dental use only.**

Instructions in written form, oral or information relayed during Producers presentation result from actual level of dental technique and technology. It should be treated as obligatory information. Information's obtained from third persons (due to their different level of authorization) do not release the user from personal check whether the preparation is intended to the designated application. Due to the fact that the application takes place without the possibility of control from the producer's side; therefore it limits the producers liability to the value of provided material.

We shall be very grateful for all comments related to products, please mail it to our address.

**Zhermapol Sp. z o.o.** Has established Full Quality Assurance System certified to ISO 9001 / ISO 13485 and under surveillance by **Notified Body Number 0120, SGS United Kingdom Ltd. Systems and Services Certification.** Product has CE mark.

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