elite hard relining

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elite hard relining

Rigid resin for direct relines.
Self-curing

Colour
T4 - pink

Composition
Self-curing acrylic resin made up of:
powder: polymethyl methacrylate
liquido: N-butyl methacrylate, ethyl methacrylate, N,N dimethyl-p-toluidine (1-1.5%)

Indications
Direct rigid relining of total or partial prostheses

Warnings
As for all resins based on methacrylate, sensitivity of the skin to the material may cause allergic reactions. If these conditions arise, suspend the use of the product.

Instructions for use

Preparation

Roughen the surface in contact with the mucous membrane with a diamond bur having a medium-large grain. Spread on the two coats of Elite Hard Relining Conditioner and wait for 2-3 minutes. In this phase, get the patient to wet his/her lips with saliva or, better, apply an extremely thin layer of FU liquid Vaseline on the mucous membrane and on the lips.

Dosing and mixing

Pour the following proportions or multiples into the mixing bowl: 1.4 ml. of liquid (1.22 g) / 1 measure of powder (2.2 g). Mix the powder and liquid with a spatula for 10 sec.

Application

Distribute the material across the surface treated previously, with a spatula. Insert in the patient's mouth, leave to adapt to the mucous membrane for a few seconds. Functionalize by carrying out the correct chewing and swallowing movements for about 1 min. Keep the prosthesis in the mouth for another 1 - 2 minutes. Remove the recessed prosthesis. Have the patient rinse his/her mouth out thoroughly.

Curing

Curing is obtained by immersing the prosthesis in water at 50-60°C for 30 min. Ideally, use should be made of hydroflask under the same conditions and at a pressure of 2 bars.

Finishing

Remove any flash caused by an excess of material and finish with silicone tips for acrylic resins.

Risk and safety phrases for liquid

R10 Flammable
R20/21/22 Harmful by inhalation, in contact with skin and if swallowed
R33 Danger of cumulative effects
R36/37/38 Irritating to eyes, respiratory system and skin
R43 May cause sensitisation by skin contact
S28 After contact with skin, wash immediately with plenty of soap and water.
S36/37 Wear suitable protective clothing and gloves.

Storage

Store in a ventilated room protected from the light at a temperature of between 5 – 27°C out of the reach of children. Do not use after the best before date.

Classification in accordance with ISO EN 10139-1

Pack size

1 jar of powder, 60 g + 1 bottle of liquid, 40 ml + 1 bottle of conditioner, 10 ml

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