


ELITE TRANSPARENT

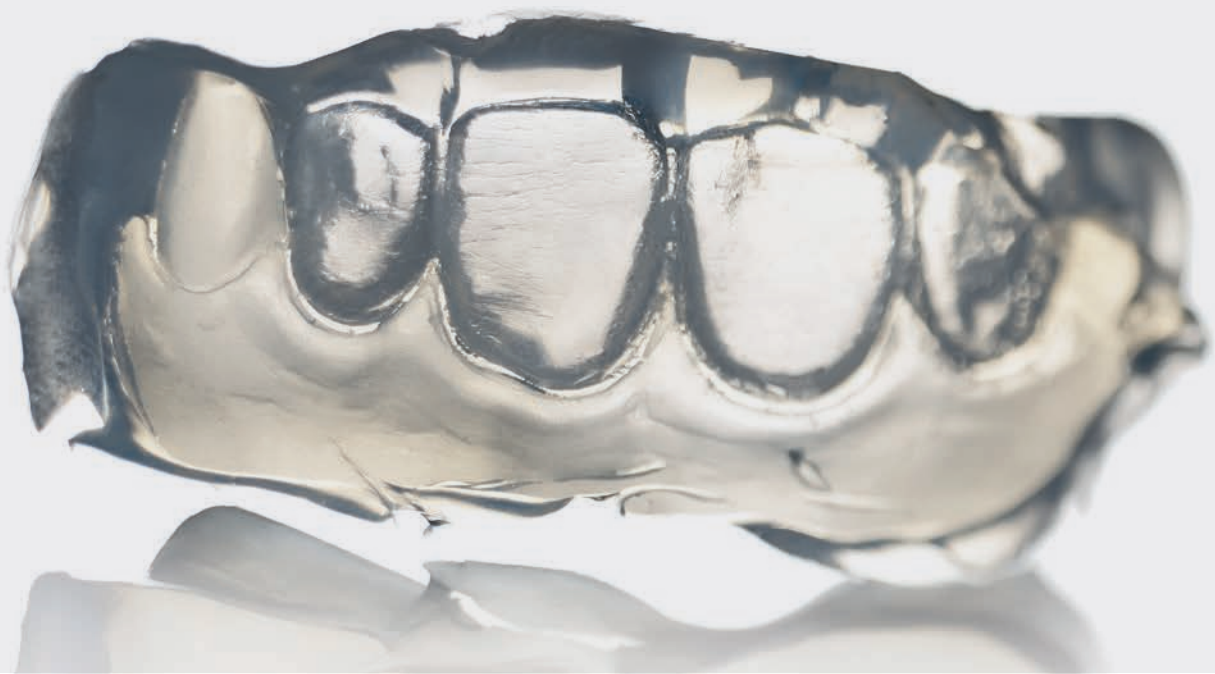

USER'S GUIDE

A-Silicone for masks for light-curing materials

Elite Transparent, simplify your work

**Elite Transparent is a transparent fluid
two-component addition silicone.**

Formulated for making transparent silicone masks
in the laboratory for composites and light-curing
resins.



1.

TEMPORARY RESTORATIONS WITH ELITE TRANSPARENT

The combined use of transparent silicones and light-curing composites for the preparation of aesthetic temporary restorations saves plenty of time without foregoing quality.

Materials used: Elite Transparent, Elite Rock.

Model



Wax-up



Application of Elite Transparent





4



Application of a transparent film to smooth the mask. Once this phase is complete, during silicone curing, the model can be placed under pressure at 2 bars



5



6



7



8

Cut back (dentine reduction to make space for the enamel)



Filling the mask with composite (enamel)



Mask repositioning and light cure



FINAL RESULT
after finishing and polishing





Technical features

Working time* (min:s)	Setting time* (min:s)	Linear dimensional change (after 24 h)	Hardness (Shore A after 1 h)	Tensile strength	Tear strength
1:30	15:00	0.18 %	72	6.5 N/mm ²	4 N/mm

*The times mentioned above are intended from the start of the mixing phase at 23° C (73° F).

Codes



Elite Transparent - A-Silicone for masks

Code	Packaging
C401600	1 x 50 ml cartridge + 6 yellow mixing tips

Find out more about related Zhermack products for masks for light-curing materials



Elite Rock

Extra-hard type 4 stone for master models in fixed prosthesis applications

For more information please visit our website: www.zhermack.com



Fulfilling your needs