**IT - Indurent Gel**

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<b>EN - Indurent Gel</b>
<b>POLYSILOXANE-SUPPORTED CATALYSTS (C-SILICONES) USE</b>
Indurent Gel is used together with the following Zhermack products: Zetaplus, Zetaplus Soft, Oranwash VL, Oranwash L, Thioxflex M, Orthogum, Zetalabor and Titanium. For applications refer to the dedicated instructions that are found in the relative packaging.
<b>WARNINGS / PRECAUTIONS</b>
- Tightly close Indurent Gel after use.
- Wear gloves and protective clothing. Avoid contacting the skin and eyes with the catalyst; in case of accidental contact with skin, wash thoroughly with soap and water. In case of contact with eyes, rinse thoroughly with running water and consult a doctor. Consult a doctor immediately if accidentally swallowed.
- Avoid staining lab coats or clothing. Take off clothing in the case of contamination.
- Not recommended for use on pregnant women.
- Stop using Indurent Gel if irritation, redness or other signs of hypersensitivity occurs during use.
<b>STORAGE AND GUARANTEE</b>
The material is guaranteed for a period of 36 months if stored correctly between 5° - 27°C (41° - 80°F).

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