# occlufast CAD

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**SOLO PER USO DENTALE**

Occlufast CAD is a vinylpolysiloxane especially made for bite registration and has been developed to be used with CAD CAM systems by optical, laser or tactile readings.  
occlufast CAD is a thixotropic, high precision and fast setting material; it is also stable and has a very high final hardness: 95Shore-A.

## PROPERTIES

Compatible with CAD CAM digital reading systems.

Thixotropic. Does not drip.

Extra rigid (95 Shore-A)

Extremely accurate

Imperceptible to the patient, who bites naturally

Dimensionally stable over time and to disinfection

Fast setting. Time in mouth <60 sec.

Can be repositioned in the mouth and on the models after setting

Easy to finish. It can be finished after setting with cutting instruments.

## APPLICATIONS

Bite registrations with all techniques which require subsequent reading with CAD CAM systems

## INSTRUCTIONS FOR USE

### Device preparation:

Assemble the device as shown in the drawings (1-2).

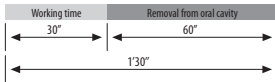
Warning: Before assembling the tip, ensure that the two components (base and catalyst) flow out evenly by applying a light pressure on the dispenser lever and extruding a small amount of material, which must be removed (3).

Subsequently, insert the mixing tip and, if required, the intraoral tip in the cartridge (4).

Apply occlufast CAD directly and without interruptions onto the occlusal surface of the teeth, before it sets the operator will ask the patient to bite with maximum intercuspitation.

Wait until it is completely hardened and remove the registration obtained. The specific consistency of occlufast CAD is thixotropic and therefore prevents the material dripping and entering into the undercut areas during registration.

## TIMES FOR CLINICAL USE

The working time is intended from the beginning of the mixing at 23° C – 73° F, the time in mouth is intended at 35° C – 95° F

**WARNING:** In sensitive subjects, vinyl polysiloxanes may cause irritation or other allergic reactions.

### **CLEANING AND DISINFECTION OF THE IMPRESSION**

After rinsing well under running water, the impression can be immediately disinfected. Ideal disinfecting is achieved by immersing the impression in **Zeta 7 solution** (Zhermack) or spraying it with **Zeta 7 spray** (Zhermack). When using other disinfectants, always follow the manufacturer's instructions.

### **CLEANING AND DISINFECTING THE DISPENSER**

To clean, disinfect or sterilize the dispenser, always follow the manufacturer's instructions. We recommend **Zeta 3** (Zhermack) disinfectant and **Zeta 2** (Zhermack) sterilisation liquid.

### **STORAGE AND WARRANTY**

The bite registration can be sent to the laboratory like a normal impression, without special precautions, provided that it is protected from mechanical stress. Store at room temperature.

occlufast CAD is guaranteed for 36 months if properly stored at temperatures from 5° to 27° C (41°-80° F).

### **WARNINGS**

- Contact with latex gloves may prevent vinylpolysiloxane from setting. Avoid direct or indirect contact with latex gloves.
- Vinylpolysiloxane is chemically resistant, so be careful not to stain your clothing.

### **TECHNICAL DATA**

Base/catalyst mixing ratio	1:1
Working time*	30 sec.
Setting time*	1' 30"
Detail reproduction	20 µm
Linear dimensional variation after 24h	- 0.05%
Hardness	95 Shore-A

\*These times are intended from the beginning of mixing at 23° C (73° F). Higher temperatures shorten, lower temperatures prolong these setting times.

**IMPORTANT REMARKS:** Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the Company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the Company and utilized by the user.

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**Zhermack S.p.A.** - Italy - via Bovazecchino, 100 - 45021 Badia Polesine (Rovigo) Italy