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HIGH-PRECISION ADDITION CURED SILICONES (POLYVINILSILOXANES) FOR THE DENTAL LABORATORY

Platinum 95 – Extra rigid setting (95 Shore-A) lab putty mixed in a 1:1 ratio

Platinum 85 / Platinum 85 TOUCH – Rigid setting (85 Shore-A) lab putty mixed in a 1:1 ratio characterised by an excellent balance between elasticity and final hardness.

Platinum 75 CAD – Vinylpolysiloxane for mixing in a 1:1 ratio with excellent consistency and workability (75 Shore A), ideal for scanning with CAD/CAM systems.

Use

- Matrix for constructing provisional prostheses.
- Addition or replacement of elements on full dentures or metal frame works.
- Matrix for use when making a removable gingival mask (Gingifast – Zhermack).
- Partial counter die in flask for partial or full-full dentures.
- Blocking in the repair of dentures.
- Occlusal regulation key for positioning in an articulator (using Platinum 75 CAD, occlusal regulation keys suitable for scanning with CAD/CAM systems can be made)
- Duplication of full dentures and gypsum models without using insulators.
- Support for working with light-curing composite resins.
- Control matrix for designing a metal framework.
- Ceramic moulded on articulator technique.

Note: The decision to use Platinum 95, 85, 85 TOUCH or 75 CAD is at the operator's discretion, whether he believes it more appropriate to use an extremely rigid silicone or a rigid silicone with some elasticity.

INSTRUCTIONS FOR USE

Dosage:

Take equal parts of the two components from the tubs using exclusively the dosing spoons provided (1). **Do not swap over the dosing spoons.**

Mixing / use:

Mix the Base and Catalyst vigorously for about 30 sec. until a uniformly coloured putty free of streaks is obtained (2).

Then start to model the putty as required. The working time, including the mixing phase, is about 2 min. (1' for Platinum 75 CAD and Platinum 85 TOUCH) at room temperature (3).

The material sets starting from a minimum of about 6 min. (4' for Platinum 75 CAD and Platinum 85 TOUCH)** The prosthesis may then be removed (4).

WARNINGS / PRECAUTIONS

The 1:1 proportions must be strictly respected to obtain the correct times and avoid altering the products' characteristics. A higher dose of the catalyst does not reduce the setting time. Close the tubs immediately after use, take care not to swap the lids, or dosing spoons.

The setting of addition silicones may be inhibited by latex gloves. We recommend the use of vinyl gloves or thorough hand washing. The surfaces with which the silicone comes into contact must be cleaned, degreased and dried.

STORAGE

Shelf life of 24 months if stored at a temperature between 5° and 27°C (41° - 80°F).

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TECHNICAL DATA	platinum95	platinum85	platinum85 TOUCH	platinum75 CAD
Mixing time	30"	30"	30"	30"
Working time*	2'	2'	1'	1'
Setting time*	8'	8'	7'	7'
Removal time**	6'	6'	4'	4'
Shore-A hardness (max)	95	85	85	75

*The times mentioned above are intended from the start of the mixing phase at 23°C (73°F).

** The times indicated may vary according to room temperature and mixing technique.

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SOLO PER USO DENTALE

IMPORTANT NOTES: Advice given verbally, in writing or in demonstrations of the use of our products is based on the current state of dental technique and on our know-how. It is to be considered informative and non-binding, even in relation to the rights of third parties, and does not exempt the user from personally ensuring that the product is suitable for the intended application. Use and application by the user is beyond the manufacturer's control and is therefore the user's responsibility. Any liability for damage shall be limited to the value of the goods supplied by the manufacturer and used by the user.

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