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EN INSTRUCTIONS FOR USE - INTENDED USE: Cleaner for removing alginate residues from impression trays and instruments. **PRODUCT DESCRIPTION:** Mixed with water in the specified proportions, Algitray cleaner forms a pH-neutral solution. **PACKAGING:** 1 kg tub + measure. **COMPOSITION:** Sodium citrate and co-formulants. **INDICATIONS FOR USE:** The product is intended for removing alginate residues from impression trays and instruments. The product is not intended for use directly on patients. **CONTRAINDICATIONS:** There are no known contraindications. **WARNINGS:** Avoid contact with eyes. In case of contact, rinse with plenty of water. **PRECAUTIONS:** Dispose of the product according to local regulations. **SIDE EFFECTS:** No side effects have been reported.

STEP-BY-STEP INSTRUCTIONS:

1. Use protective gloves, face mask, eyewear and work clothes.
2. Add 100 g of Algitray (4 measures) to 1 litre of water and stir to facilitate dissolution of the powder. In case of incomplete dissolution, there may be a reduction of the cleaning action.
3. Remove as much alginate from the impression trays or instruments as possible.
4. Immerse the impression trays or instruments in the solution.
5. Leave them to soak in static bath for 15-30 minutes or in ultrasonic bath for 5-10 minutes at room temperature.
6. Remove the impression trays or instruments from the solution. Brush off all remaining traces of alginate then rinse under running water.
7. Leave the impression trays or instruments to dry before disinfecting and sterilising them ready for the next use.
8. Replace the solution when the cleaning action is compromised.

STORAGE AND STABILITY: Store the product at a temperature between 5 °C and 27 °C. Do not use the product after the expiry date printed on the packaging. **IMPORTANT NOTES:** The instructions must be respected, irrespective of possible information given in any other way, including during demonstrations. It is the operator's responsibility to ensure that the product is suitable for the application in question. The Company is not liable for possible damage, including to third parties, deriving from failure to respect the instructions or inappropriate use of the product and in any case within the limits of the value of the products supplied. **PROFESSIONAL USE ONLY**

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Zhermack
algitray

specific for trays and instruments

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