elite P&P

hydrophilic

vinylpolysiloxane (addition silicone)
impression material

putty soft normal

putty soft fast

light body normal

light body fast

regular body normal

Last Update-2020-08

U117090-2020-08

 **Zhermack S.p.A.**

Via Bovazecchino, 100 | 45021 Badia Polesine (RO) Italy
T +39 0425 597611 | F +39 0425 53596
info@zhermack.com | www.zhermack.com

elite P&P

hydrophilic

Suggested Techniques	Viscosity	Recommended materials
2-Step Technique	2	Elite P&P Putty Normal + Light Normal
2-Step Technique	2	Elite P&P Putty Fast + Light Fast
1-Step Technique	2	Elite P&P Putty Normal + Light Normal
2-Step Technique	2	Elite P&P Putty Normal + Regular Normal
1-Step Technique	1	Elite P&P Putty Normal + Regular Normal
1-Step Monophase	1	Elite P&P Regular Normal

Packaging

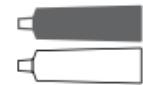
Jars

C206000	putty normal set	1x 250 ml Base jar + 1x 250 ml Catalyst jar	
C206001	putty fast set	1x 250 ml Base jar + 1x 250 ml Catalyst jar	

Cartridges


C206010	light body normal set	2x 50 ml cartridges (Base+Catalyst) + 6 yellow mixing tips	
C206011	light body fast set	2x 50 ml cartridges (Base+Catalyst) + 6 yellow mixing tips	
C206012	regular body normal set	2x 50 ml cartridges (Base+Catalyst) + 6 yellow mixing tips	

Tubes



C206020	light body normal set	1x 60 ml Base tube + 1x 60 ml Catalyst tube	
C206021	regular body normal set	1x 60 ml Base tube + 1x 60 ml Catalyst tube	

elite P&P

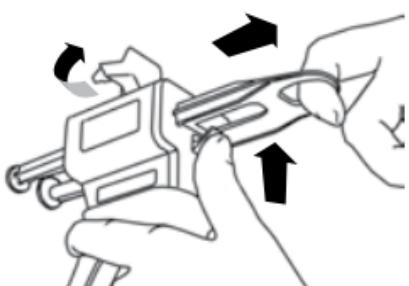
1

2

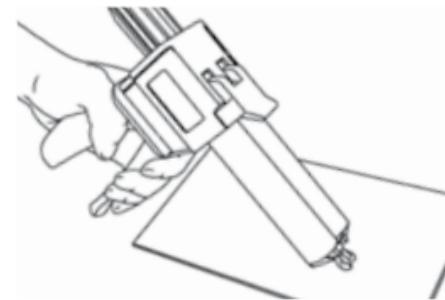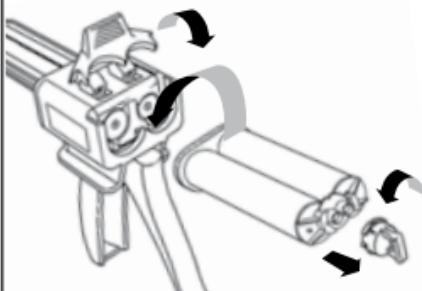3

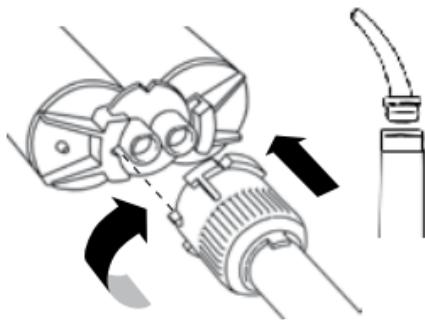4

HYDROPHILIC VINYL POLYSILOXANE (A-SILICONE) FOR HIGH-PRECISION IMPRESSIONS

USE

Elite P&P Putty Soft Normal Set: Vinyl polysiloxane for first impressions with elastic final consistency and normal Set time. Recommended for the two-phase technique with two viscosities (two-stage) or one phase with two viscosities (single-stage).

Elite P&P Putty Soft Fast Set: Vinyl polysiloxane for first impressions with elastic final consistency and fast Set time. Recommended for the two-phase technique with two viscosities (two-stage).

Elite P&P Light Body Normal Set: Hydrophilic vinyl polysiloxane with low viscosity and normal Set time. Recommended for the two-phase technique with two viscosities (two-stage) or one phase with two viscosities (single-stage).

Elite P&P Light Body Fast Set: Hydrophilic vinyl polysiloxane with low viscosity and fast Set time. Recommended for the two-phase technique with two viscosities (two-stage).

Elite P&P Regular Body Normal Set: Water compatible vinyl polysiloxane with medium viscosity and normal Set time. Suitable for the one phase technique with one viscosity, recommended for the two-phase technique with two viscosities (two-stage) or one phase with two viscosities (single-stage). Suitable also to record an impression which will be used to rebase a removable denture.

IMPRESSION TRAY: PREPARATION AND ADHESIVES

All standard and/or special trays can be used, provided they are rigid and non-deformable. The range of **Zhermack's Hi-Tray** impression trays are especially recommended.

The adhesion of the impression material to the impression tray is a crucial phase in which errors must be avoided, above all when removing from the mouth.

UNIVERSAL TRAY ADHESIVE: Adhesive for standard metal and plastic impression trays or individual resin impression trays.

Elite P&P Putty

DOSAGE AND MIXING:

WARNING: do not handle with latex gloves, carefully wash the hands before handling.

- Take equal amounts of the two components from the tins, only using the respective dosing spoons. **USE THE SPOONS THAT ARE THE SAME COLOUR AS THE TIN. DO NOT SWAP.**
- Mix equal proportions of Elite P&P Putty Soft Base and Catalyst, until creating a mass of uniform colour, without streaks (mixing time: around 30 sec.).

WARNINGS:

- The proportions must be observed in order to ensure the correct Set times; a higher dosage of catalyst does not speed up the Set times.
- Close the tins immediately after use. Do not swap the caps and dosing spoons.
- The Set of vinyl polysiloxane is inhibited by contact with latex gloves. Avoid direct or indirect contact with latex gloves.

The hands should be washed thoroughly to remove all traces of contaminant if latex gloves had been used previously.

The use of vinyl gloves is allowed.

- Vinyl polysiloxanes are chemically resistant, therefore avoid staining overcoats and clothing.

Elite P&P Light

Elite P&P Regular

60ML TUBE VERSION

MIXING:

Squeeze out equal amounts of the two components onto the mixing pad.

Carefully mix with a metal spatula so as to reach a uniform colour, for around 30 seconds

After use, close the tubes again, making sure not to swap the caps.

Elite P&P Light

Elite P&P Regular

50ML CARTRIDGE VERSION

PREPARING THE DEVICE:

Set up the device as illustrated in the drawings (1-2).

Warning: before fitting the mixing tip, check that that the two components (base and catalyst) are released uniformly, by pressing lightly on the dispenser lever and squeezing out a small amount of material, which must be removed (3).

Then fit the mixing tip on the cartridge and the intraoral tip, if used (accessory) (4). After use, leave the tip on, as this will act as a cap until subsequent use.

WARNING: In sensitive subjects, vinyl polysiloxanes may cause irritation or other allergic reactions.

CLEANING AND DISINFECTING THE IMPRESSION

After rinsing well under running water, the impression can be immediately disinfected. Ideal disinfecting is achieved by immersing the impression in **Zeta 7 solution** or spraying it with **Zeta 7 spray** (Zhermack). When using other disinfectants, always follow the manufacturer's instructions.

CASTING MODELS

For best results, wait a minimum 30 minutes before casting the impression. The impression remains dimensionally stable for a minimum of 7 days and a maximum of 14 days, as long as it is kept at ambient temperature. Elite P&P is compatible with all the best plasters available on the market, and the polyurethane and epoxy resins for models. It is recommended to use extra hard gypsum for dies: **Elite Rock/Elite Rock Fast (Zhermack)** and hard gypsum for models: **Elite Model/Elite Model Fast (Zhermack)**.

CLEANING AND DISINFECTING THE DISPENSER

To clean, disinfect or sterilize the dispenser, always follow the manufacturer's instructions.

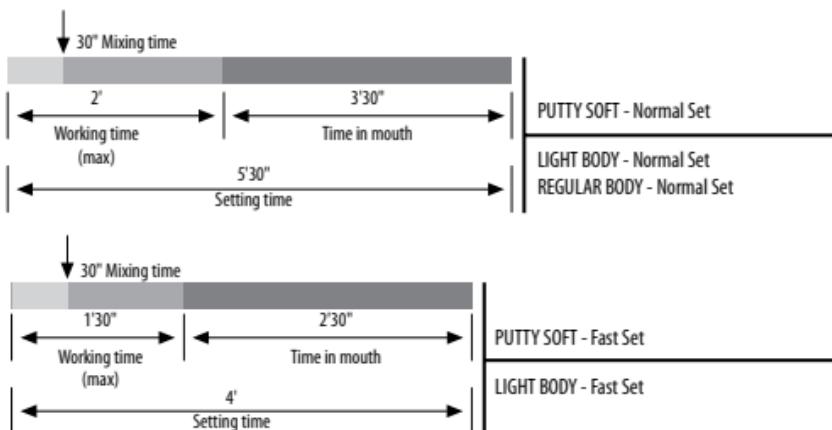
STORAGE AND WARRANTY

The impressions made with Elite P&P must be kept at ambient temperature. Elite P&P impression materials are guaranteed for 36 months from the date of production, if stored correctly between 5° and 27° C / 41°- 80°F.

GALVANISATION

The impressions made with Elite P&P can be galvanised with copper or silver.

TIMES FOR CLINICAL USE

TECHNICAL DATA

Elite P&P	Light Body Normal Set		Light Body Fast Set		Regular Body Normal Set		Putty Soft Normal Set		Putty Soft Fast Set	
ISO 4823	Type 3 Light-bodied Consistency		Type 3 Light-bodied Consistency		Type 2 Medium-bodied Consistency		Type 0 Putty Consistency		Type 0 Putty Consistency	
Base: Catalyst mixing ratio	1:1		1:1		1:1		1:1		1:1	
Mixing time	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix	30"	30"	30"	30"	30"
Total working time*	2'		1'30"		2'		2'		1'30"	
Set time*	5'30"		4'		5'30"		5'30"		4'	
Compressive deformation (min – max)	3 – 5%		3 – 5%		3 – 5%		1 – 3%		1 – 3%	
Elastic recovery	> 99.5%		> 99.5%		> 99.5%		> 99.0%		> 99.0%	
Dimensional stability after 24 hours	< 0.20%		< 0.20%		< 0.20%		< 0.20%		< 0.20%	
Shore A hardness	45±2		45±2		45±2		55±2		55±2	

*The working time is considered from the beginning of mixing at 23°C – 73°F, the time in mouth is considered at 35°C – 95°F

IMPORTANT REMARKS: Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the Company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the Company and utilized by the user.

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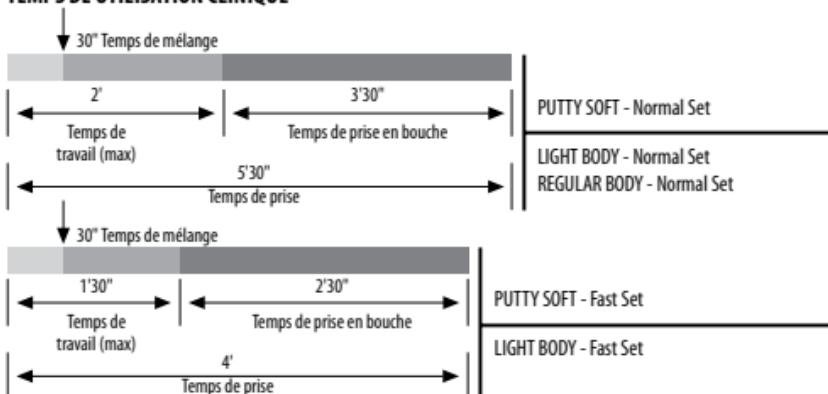
INFORMATIONS TECHNIQUES

Elite P&P	Light Body Normal Set	Light Body Fast Set*	Regular Body Normal Set*	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Rapport de mélange Basé : catalyseur	1:1	1:1	1:1	1:1	1:1
Temps de mélange	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix
Temps total et d'utilisation *	2'	1'30"	2'	2'	1'30"
Temps de prise *	5'30"	4'	5'30"	5'30"	4'
Déformation à la compression (min – max)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Réprise élastique	> 99,5%	> 99,5%	> 99,5%	> 99,0%	> 99,0%
Stabilité dimensionnelle 24 heures	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%
Dureté Shore-A	45±2	45±2	45±2	55±2	55±2

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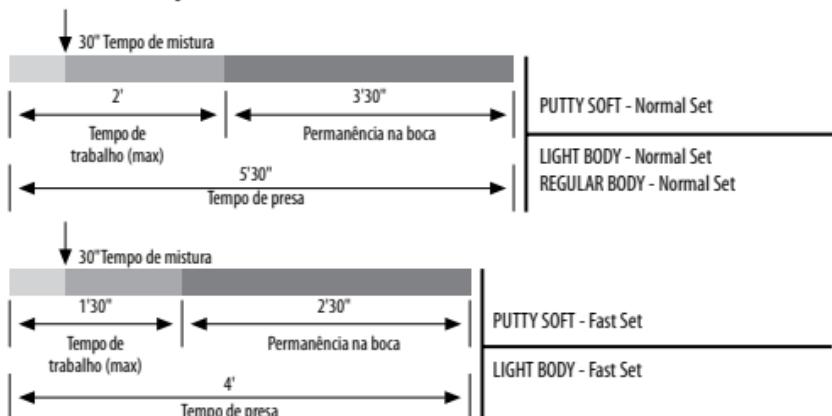
Os moldes de Elite P&P podem ser galvanizados com cobre ou prata.

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Elite P&P	Light Body Normal Set	Light Body Fast Set	Regular Body Normal Set	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Relação de mistura Base: Catalisador	1:1	1:1	1:1	1:1	1:1
Tempo de mistura	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix
Tempo total de trabalho*	2'	1'30"	2'	2'	1'30"
Tempo de aplicação*	5'30"	4'	5'30"	5'30"	4'
Deformação na compressão (min. – máx.)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Recuperação elástica	> 99,5%	> 99,5%	> 99,5%	> 99,0%	> 99,0%
Estabilidade dimensional após 24 horas	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%
Dureza Shore-A	45±2	45±2	45±2	55±2	55±2

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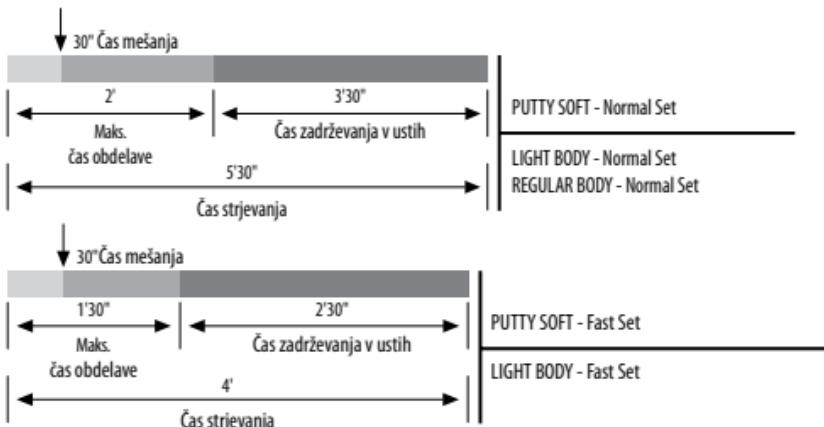
Elite P&P	Light Body Normal Set	Light Body Fast Set	Regular Body Normal Set	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Razmerje mešanja osnova:katalizator	1:1	1:1	1:1	1:1	1:1
Čas mešanja	in tube 30"	I in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix
Skupni čas obdelave	2'	1'30"	2'	2'	1'30"
Čas strjevanja	5'30"	4'	5'30"	5'30"	4'
Kompresivna deformacija (najmanjša-največja)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Ponovna pridobitev elastičnosti	> 99,5%	> 99,5%	> 99,5%	> 99,0%	> 99,0%
Dimenzionalna obstojnost po 24 urah	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%
Trdota Shore-A	45±2	45±2	45±2	55±2	55±2

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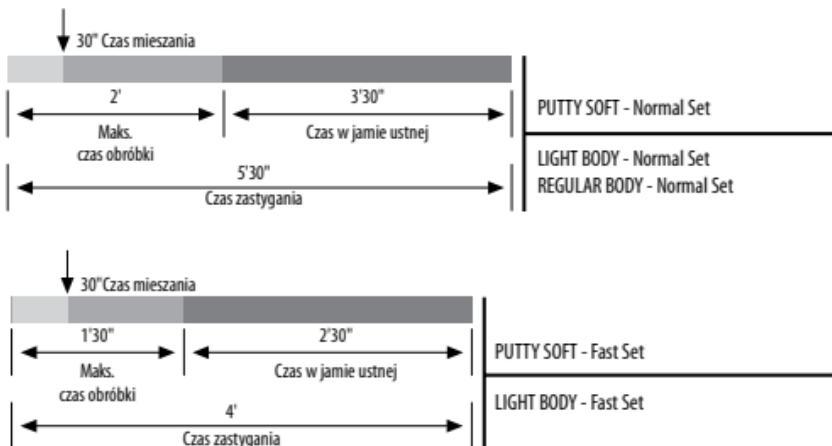
Elite P&P	Light Body Normal Set	Light Body Fast Set	Regular Body Normal Set	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Proporcja mieszania podłożka:katalizatora	1:1	1:1	1:1	1:1	1:1
Czas mieszania	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix
Calkowity czas obróbki*	2'	1'30"	2'	2'	1'30"
Czas twardnienia *	5'30"	4'	5'30"	5'30"	4'
Odkształcenie przy ściskaniu (min – maks)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Powrót elastyczny	> 99.5%	> 99.5%	> 99.5%	> 99.0%	> 99.0%
Stabilność wymiarowa po 24 godzinach	< 0.20%	< 0.20%	< 0.20%	< 0.20%	< 0.20%
Twardość w skali Shore A	45±2	45±2	45±2	55±2	55±2

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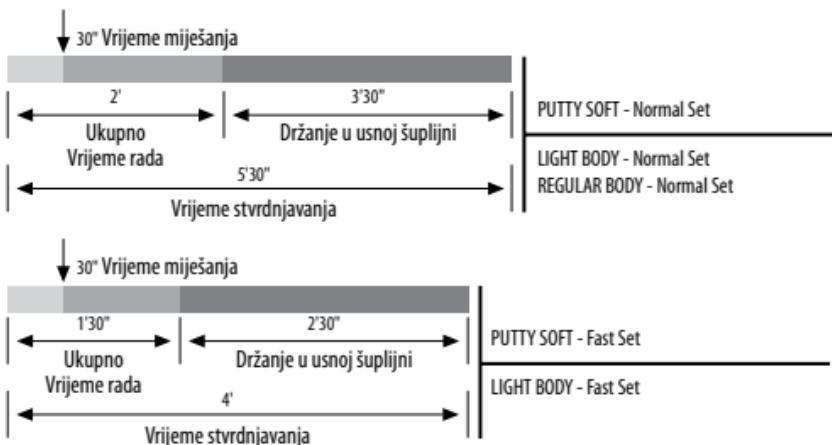
VRIJEME KLINIČKE UPORABE

TEHNIČKI PODACI

Elite P&P	Light Body Normal Set	Light Body Fast Set	Regular Body Normal Set	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Omjer miješanja baza:katalizator	1:1	1:1	1:1	1:1	1:1
Vrijeme miješanja	in tube 30°	in cartridge automix	in cartridge automix	30°	30°
Ukupno vrijeme rada*	2'	1'30"	2'	2'	1'30"
Vrijeme stvrdnjavanja*	5'30"	4'	5'30"	5'30"	4'
Tlačno naprezanje (min – max)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Elastični povrat	> 99,5%	> 99,5%	> 99,5%	> 99,0%	> 99,0%
Dimenzijska stabilnost nakon 24 sata	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%
Tvrdoća Shore-A	45±2	45±2	45±2	55±2	55±2

*Vrijeme rada određeno je od početka miješanja pri 23°C – 73°F, vrijeme u ustima određeno je pri 35°C – 95°F.

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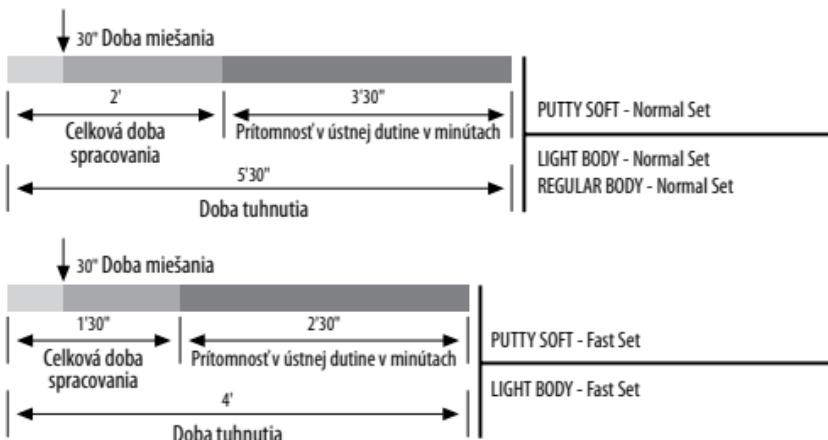
Elite P&P	Light Body Normal Set		Light Body Fast Set	Regular Body Normal Set		Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency		Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency		Type 0 Putty Consistency	Type 0 Putty Consistency
Pomer miešania Základ:Katalyzátor	1:1		1:1	1:1		1:1	1:1
Doba miešania	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix	30"	30"
Celková doba spracovania*	2'	1'30"	2'	2'	2'	2'	1'30"
Doba prílnutia*	5'30"	4'	5'30"	5'30"	5'30"	5'30"	4'
Deformácia pri tlaku (min - max)	3 - 5%	3 - 5%	3 - 5%	3 - 5%	3 - 5%	1 - 3%	1 - 3%
Elastická obnova	> 99,5%	> 99,5%	> 99,5%	> 99,5%	> 99,5%	> 99,0%	> 99,0%
Rozmerová stabilita po 24 hodinách	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%	< 0,20%
Tvrdosť Shore-A	45±2	45±2	45±2	45±2	45±2	55±2	55±2

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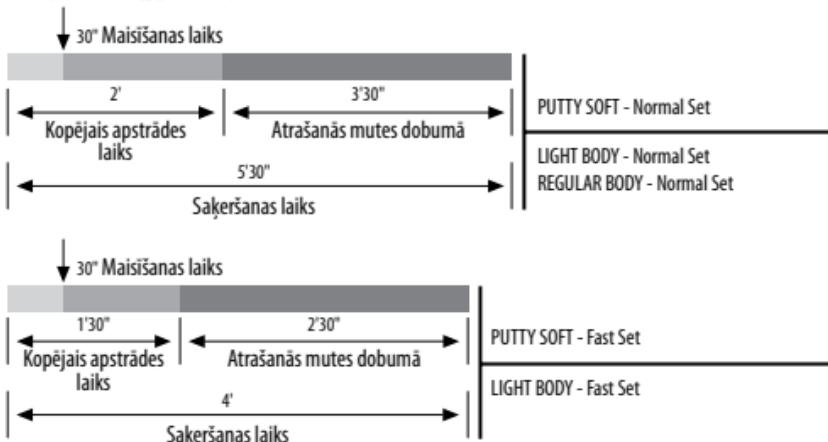
Elite P&P veiktos nospiedumus var galvanizēt ar varu vai sudrabu

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TEHNISKIE DATI

Elite P&P	Light Body Normal Set		Light Body Fast Set		Regular Body Normal Set		Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency		Type 3 Light-bodied Consistency		Type 2 Medium-bodied Consistency		Type 0 Putty Consistency	Type 0 Putty Consistency
Bāzes attiecība pret katalizatoru	1:1		1:1		1:1		1:1	1:1
Mixing time	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix	30"	30"	30"
Kopējais darbības laiks	2'		1'30"		2'		2'	1'30"
Societēšanas laiks*	5'30"		4'		5'30"		5'30"	4'
Spiedes deformācija (min.-maks.)	3 – 5%		3 – 5%		3 – 5%		1 – 3%	1 – 3%
Formas elastīgā atjaunošanās pēc saspiešanas	> 99.5%		> 99.5%		> 99.5%		> 99.0%	> 99.0%
Izmēru stabilitāte pēc 24 stundām	< 0.20%		< 0.20%		< 0.20%		< 0.20%	< 0.20%
A cietība pēc Šora	45±2		45±2		45±2		55±2	55±2

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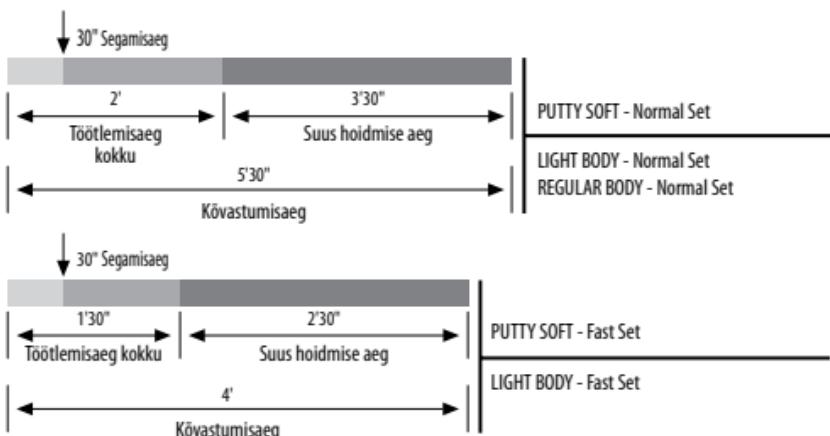
TEHNILISED ANDMED

Elite P&P	Light Body Normal Set	Light Body Fast Set	Regular Body Normal Set	Putty Soft Normal Set	Putty Soft Fast Set
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 2 Medium-bodied Consistency	Type 0 Putty Consistency	Type 0 Putty Consistency
Aluse-katalüsaatori segamissuhe	1:1	1:1	1:1	1:1	1:1
Segamisaeg	in tube 30"	in cartridge automix	in cartridge automix	in tube 30"	in cartridge automix
Kogu tööaeg*	2'	1'30"	2'	2'	1'30"
Tahkumisaeg*	5'30"	4'	5'30"	5'30"	4'
Kompressiivne deformatsioon (min. – maks.)	3 – 5%	3 – 5%	3 – 5%	1 – 3%	1 – 3%
Elastuse taastumine	> 99.5%	> 99.5%	> 99.5%	> 99.0%	> 99.0%
Mõõtmete stabiilsus 24 h järel	< 0.20%	< 0.20%	< 0.20%	< 0.20%	< 0.20%
Shore A kõvadus	45±2	45±2	45±2	55±2	55±2

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Zhermack S.p.A. - Italy - via Bovazecchino, 100 - 45021 Badia Polesine (Rovigo) Italy