making work easy

www.renfert.com

SYMPRESS

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Made in Germany

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Contents

en

1	Introduction	2
1.1	Symbology	2
1.2	EU Conformity	2
2	Safety	3
2.1	Proper Use	3
2.2	Ambient conditions for safe operation	3
2.3	Ambient conditions for storage and transport	3
2.4	Hazard and Warning Information	3
2.4.1	General Information	3
2.5	Information about electromagnetic compatibility (EMC)	4
2.6	Authorised Individuals	4
2.7	Liability Exclusion	4
3	Product Description	4
3.1	General Description	4
3.2	Assembly and Controls	5
3.3	Standard Delivery	6
3.4	Accessories	6
4	Commissioning	6
4.1	Setup	6
4.1.1	Benchtop Unit	6
4.1.2	Wall Mounting	6
4.1.3	Setup Recommendations	6
4.2	Connection and Starting	6
4.3	Remove Shipping Restraint	6
5	Operation	7
5.1	Cartridge Installation and Replacement	7
5.2	Extrusion	7
5.3	Timer Function	8
5.3.1	Timer Function ON/OFF	8
5.4	Fill Level Indicator and Remaining Amount	8
5.4.1	Fill Level Indicator	8
5.4.2	Remaining Amount Detection	8
6	Cleaning / Maintenance	8
6.1	Cleaning the Interior	9
6.2	Pressure Disc Replacement	9
6.3	Unit Cover Replacement	9
6.4	Fuse Replacement	9
6.5	Spare Parts	9
7	Troubleshooting	10
8	Technical Specifications	11
9	Warranty	11
10	Disposal Information	11
10.1	Disposing of Consumables	11
10.2	Instrument Disposal	11
10.2.1	Disposal instructions for countries in the EU	11

1 Introduction

1.1 Symbology

The following symbols are employed in these instructions and on the unit itself:

 **Follow the Instructions for Use!**

 **Caution**

Serious risk of injury!

 **Electrical current**

This indicates a hazard due to electrical current.

 **Attention**

Failure to observe the associated information can result in damage to the unit.

 **Note**

This provides the operator with useful information to make working with the unit easier.

 **i**

The operating instructions can be found in electronic format on our website.

Visit www.renfert.com/p915. Enter the following item number: 6000 0000.

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Please observe the operating time.

After no more than 2 minutes of operation, you must pause for at least 5 minutes.

After immediate processing of 3 complete cartridges (approx. 30 impressions, middle-sized impression tray) a 2 hours quiescent period should be observed. During this time, the unit may remain connected.

 **CE**

The EU regulations applicable to the product are adhered to.

Within the EU, this device is subject to the provisions of the WEEE directive.

Manufacturer

Date of manufacture

 **MD**

Medical device

 **Rx only** To be used by specialist staff.

► **List, particular attention should be paid**

- List
 - Subordinate List

⇒ Instructions / appropriate action / input / operational sequence:

You will be asked to carry out the action in a specified order.

◆ Result of an action / reaction of the device / reaction of the program:

The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

1.2 EU Conformity

Renfert GmbH hereby declares that this product:

SYMPRESS

Item number: 6000 0000, 1000, 2000, 3000

complies with the following European regulations:

Regulation (EU) 2017/745

2 Safety

⚠ Using these operating instructions as a starting point, instruct all operators of the unit with regard to the area of application, the possible hazards during operation, and the proper operation of the unit.

Please have these operating instructions readily available for the operators.

i In case of complaints and events in connection with the medical device, please inform Renfert immediately. Renfert will investigate the matter and report serious incidents to the authorities in the Member States concerned.

2.1 Proper Use

The SYMPRESS is designed solely for the extrusion of 2-component precision impression materials. With the SYMPRESS it is possible to use 5:1 cartridges or pouches in support cartridges from various manufacturers.

Dynamic mixing nozzles with hex drive are used to mix the material.

⚠ Always comply with all safety and processing instructions provided by the manufacturer!

⚠ Only the mixing nozzles specified by the manufacturer may be employed.

2.2 Ambient conditions for safe operation

Safe operation of the unit can only be ensured under the following ambient conditions:

- Indoors;
- Up to an altitude of 2,000 m above sea level;
- At an ambient temperature range between 5 - 40 °C [41 - 104 °F] *);
- At a maximum relative humidity of 80 % at 31 °C [88 °F], dropping in a linear manner to 50 % relative humidity at 40 °C [104 °F] *);
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value;
- Under contamination level 2 conditions;
- Under over-voltage category II conditions.

*) Between 5 – 40 °C [41 – 104 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 – 40 °C [88 – 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

2.3 Ambient conditions for storage and transport

Ensure the following ambient conditions during storage and transport:

- Ambient temperature - 20 – + 60 °C [- 4 – + 140 °F],
- Maximum relative humidity 80 %.

2.4 Hazard and Warning Information

2.4.1 General Information

- ▶ Only to be used indoors. The unit is only intended for dry operation and should not be used or stored outdoors or in wet conditions.
- ▶ The device may only be operated using a mains cable with the country-specific plug system.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Pull out the mains plug to disconnect the equipment from the mains supply.
- ▶ Check connection cables (such as power cords), tubes and housing (i.e. the key pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing.
- ▶ Do not operate equipment with damaged connection cables, hoses or housing parts or other defects!

- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ Always unplug the unit from the wall outlet before beginning any work on the unit's electrical components.
- ▶ Note any hazard or warning information provided by the material manufacturer.
- ▶ Do not operate in an explosion-prone area.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with.
For Germany these are MPBetreibV in relation with DIN EN 62353 (VDE0751).
- ▶ Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

2.5 Information about electromagnetic compatibility (EMC)

Information about electromagnetic compatibility (EMC) can be found in a separate section at the end of these instructions.

2.6 Authorised Individuals

This product may not be used by minors under the age of 14.

Only properly trained individuals may operate and service the unit.

Any repairs not specifically described in these operating instructions may only be performed by a qualified electrician or authorized dealers.

2.7 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those cited in the operating instructions;
- ▶ The product is altered in any way other than those alterations described in the operating instructions;
- ▶ The product is repaired by other than an authorised facility or if any but Renfert OEM parts are employed;
- ▶ The product continues to be employed, despite obvious safety faults or damage;
- ▶ The product is subjected to mechanical impacts or is dropped.
- ▶ We do not accept any liability for damage caused by mixing tips, foil bags or cartridges splitting.

3 Product Description

3.1 General Description

The SYMPRESS unit for mixing impression materials facilitates extrusion and mixing of 2-component dental impression materials.

5:1 cartridges or foil bags in cartridge holders from different manufacturers can be used in the unit.

Only mixing nozzles should be used which are designated to the impression material by the manufacturer.

3.2 Assembly and Controls

en

- | | |
|---|---|
| 1 Fill level indicator | 11 Spindles |
| 2 "Impression tray" advance key
(fast extrusion speed) | 12 Catch |
| 3 "Dispenser" advance key
(slow extrusion speed) | 13 Hexagonal end |
| 4 Retract key | 14 Cartridge bed |
| 5 Yellow LED, residual level indicator | 15 --- |
| 6 Green LED, unit ON | 16 Mains switch |
| 7 Red LED, unit cover open | 17 Fuse |
| 8 Unit cover | 18 Mains connection |
| 9 Pressure discs | 19 Power cord |
| 10 Spindle covers | 20 Cartridge / Cartridge with tubular bag |
| | 21 Mixing nozzle |

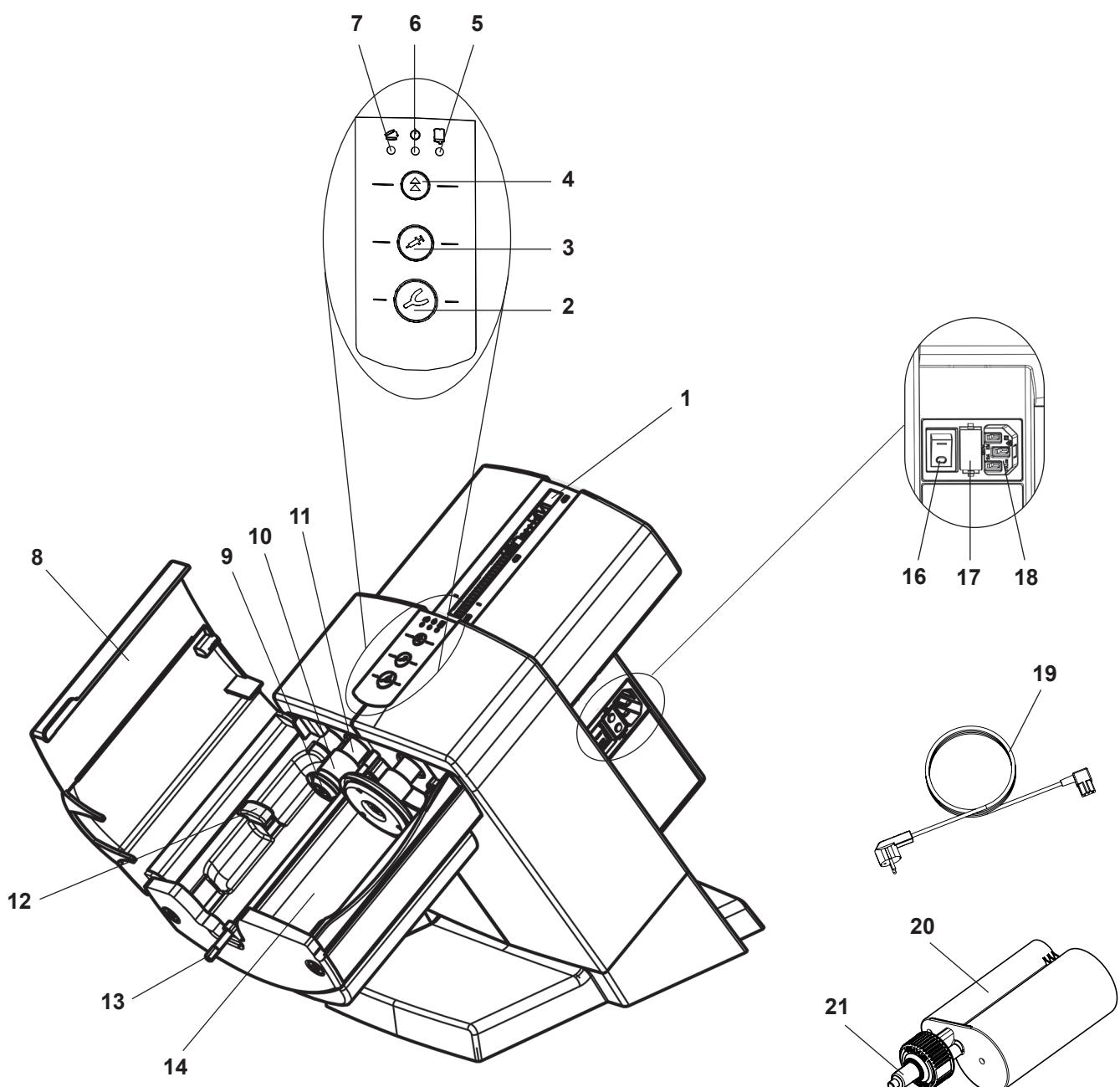
Fig. 1

3.3 Standard Delivery

- 1 SYMPRESS mixing unit
- 1 Power cord
- 1 Operating instructions

3.4 Accessories

- 6000 0100 Wall mounting kit

4 Commissioning

4.1 Setup

The equipment can be used as a benchtop unit, although it is also suitable for wall mounting.

4.1.1 Benchtop Unit

Select a stable underlay on which to set the unit up.

4.1.2 Wall Mounting

With the aid of the wall mounting kit (refer to the accessories), the unit can be securely mounted on a wall.

The mounting instructions are included in the mounting kit.

4.1.3 Setup Recommendations

Operate the unit at room temperature 18 - 24 °C [64 – 75 °F].

When setting up the unit, please note the following:

- Do not place the unit under a heat source.
- Do not place the unit in front of open windows.
- Do not set the unit up where it will be subjected to direct sunlight.
- Do not expose the unit to high humidity.

4.2 Connection and Starting

The equipment may only be operated if the information on the rating plate conforms to the specifications of your local mains power supply.

- ⇒ Connect the provided power cord (19, Fig. 1) to the mains connection socket (18).
- ⇒ Plug the power cord into the wall outlet.
- ⇒ Switch the unit on at the mains power switch (16).

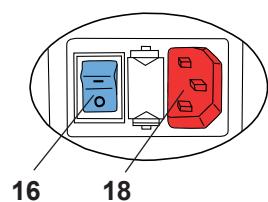
Fig. 2

4.3 Remove Shipping Restraint

The shipping restraint must be removed before the unit may be operated:

- ⇒ Connect the equipment as described in Chapter 4.2 and switch it on.
- ⇒ Close the unit cover (8, Fig. 1).
- ⇒ Press the retract key (4).
 - ♦ The pressure plates (9, Fig. 1) retract to their home position where they stop.
- ⇒ Open the unit cover.
- ⇒ Remove the shipping restraint (23).

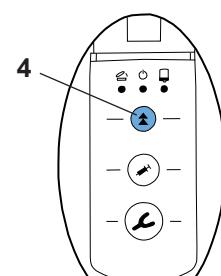
Fig. 3

The unit is now ready for use.

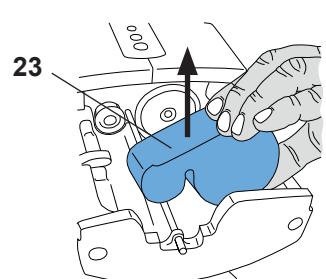
Fig. 4

5 Operation

5.1 Cartridge Installation and Replacement

A cartridge can only be installed or replaced if both pressure plates (9, Fig. 1) are in their home position.

⇒ Close the unit cover (8, Fig. 1).

⇒ Press the retract key (4).

- ◆ The pressure plates retract to their home position and stop at the end automatically.

i Just before the end, the retraction speed is reduced. Do not open the equipment cover until the pressure plates have come to a complete stop

i If the pressure plates are stopped by pressing one of the advance keys, it will not resume automatically. Press the retract key again to resume the retract motion.

⇒ Open the unit cover (8, Fig. 1).

⇒ Install the cartridge (20) into the equipment. Please make sure that the cartridge is fitted just below the retention lug (12).

⇒ Close the unit cover (8, Fig. 1).

The cover will only close if the cartridge has been properly installed. The red "Equipment cover" LED will remain on until the cover has been properly closed (7, Fig. 1).

⇒ To prepare and properly level the cartridge systems, follow the instructions provided by the impression material manufacturer.

! **Use only mixing nozzles recommended by the impression material manufacturer. Use of any other mixing nozzles may cause the equipment to malfunction.**

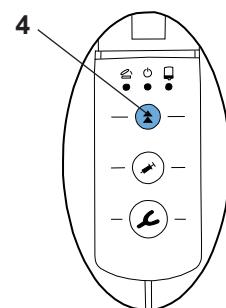
Fig. 5

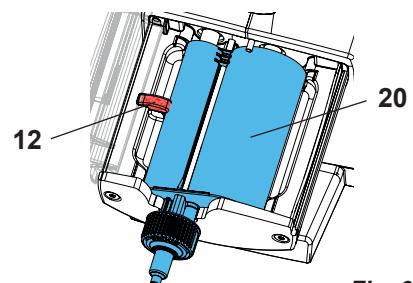
Fig. 6

5.2 Extrusion

⇒ Prepare the equipment as described in Chapter 5.1.

⇒ Press and hold down the advance key (2) or (3):

- Impression tray (2), fast extrusion speed

or

- Dispenser (3), slow extrusion speed

- ◆ The mixer shaft (13, Fig. 1) starts to turn and threads its way automatically into the mixing nozzle (21, Fig. 1), where it stops.

- ◆ The pressure plates advance at a high speed until they reach the impression material.

- ◆ Once they make contact with the impression material, the equipment automatically switches to the selected extrusion speed and the mixing shaft starts to turn.

⇒ Fill the impression tray or dispenser.

⇒ Release the advance key as soon as the desired amount of material has been mixed.

- ◆ The pressure plates retract with a short relief stroke to prevent any additional impression material from being extruded.

i The pressure plates will only move if the cover is closed.

i An audible noise may be heard while threading the mixer shaft into the mixing nozzle. This is perfectly normal.

i The first 5 cm of material from a newly installed cartridge or from an again inserted used cartridge should be discarded.

Please note the recommendations from the impression material manufacturer!

i If the cartridge is emptied during an extrusion, the pressure plates automatically return to their home position so that a new cartridge can be quickly installed and a new extrusion process can start.

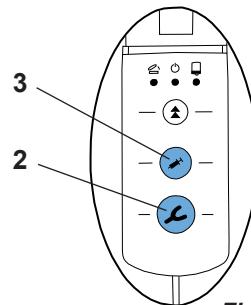
Fig. 7

5.3 Timer Function

The SYMPRESS has an integrated timer.

On delivery of the unit the timer is not activated. To activate the timer, see 5.3.1.


The activated timer starts automatically at the end of the press cycle. Timer start is indicated by a brief audible signal.

i *The timer will not start if the advance keys are briefly pressed and released without any impression material being extruded.*

The timer employs three different audible signal lengths to indicate the end of three predefined time periods:

- After 3 min.: 1 signal
- After 4 min.: 2 signals
- After 5 min.: 3 signals

The timer can be stopped by simultaneously pressing both advance keys.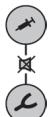

Timer stop is confirmed by a brief audible signal.

5.3.1 Timer Function ON/OFF

The overall timer function can be activated or deactivated:

To do this:

- ⇒ Switch the unit off.
- ⇒ Press and hold both advance keys.
- ⇒ While continuing to hold the advance keys, switch the unit on again:
 - Brief audible signal: Timer function is deactivated
or
 - Long audible signal: Timer function is activated.
- ⇒ Release both advance keys.

5.4 Fill Level Indicator and Remaining Amount

5.4.1 Fill Level Indicator

The fill level indicator (1) helps you to estimate the amount of impression material remaining in the cartridge.

However, after installing a new cartridge or if the cartridge is very full, the red indicator will at first not be visible in the viewing pane.

The indicator will not become visible until the pressure plates have advanced approx. 25 mm into the cartridge.

5.4.2 Remaining Amount Detection

The SYMPRESS is equipped with an electronic fill level detection. Once there is only sufficient material remaining in a cartridge to fill approx. one casting spoon, the yellow "Residual level" LED (5) goes on.

In the process, the indicator exceeds a marked remaining quantity area.

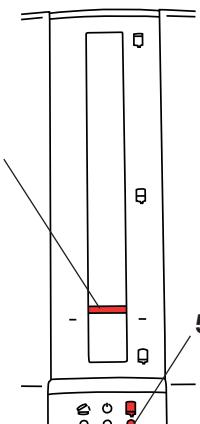
Fig. 8

6 Cleaning / Maintenance

The equipment does not contain any parts that require maintenance.

Opening the equipment other than described in the instructions is not permitted!

Never use cleansers containing solvents.

Never use superheated steam for cleaning! .

To clean the unit, merely wipe it down with a moist cloth.

6.1 Cleaning the Interior

- Cartridge bed (14, Fig. 1):

With the pressure plates (9, Fig. 1) retracted, the cartridge bed is easily accessible for cleaning.

- Pressure plates (9, Fig. 1) and spindle cover (10, Fig. 1):

If the cover is closed and with no installed cartridge, the pressure plates can be retracted to the appropriate position to allow residue to be removed from the pressure plates or the spindle covers.

! **NEVER clean the spindles (11, Fig. 1) themselves. The grease applied to spindles is necessary for the unit's proper function.**

Tip:

Remnants of impression material are most easily removed with a dry cloth or paper towel.

In addition to wiping with a moist cloth, the following cleaning agents are approved:

- Incidur Spray (Ecolab)
- Incides N wipes (Ecolab)

Ensure adequate ventilation after using these agents to prevent an explosion-prone area.

6.2 Pressure Disc Replacement

The pressure plates (9, Fig. 1) are subject to wear as a result of their rubbing against the inner cartridge wall (20, Fig. 1).

Highly worn pressure plates may cause damage to the foil bag in the cartridge holder (20, Fig. 1).

Moreover, an improperly installed cartridge may cause a crack in the pressure plates.

This is why the pressure plates are available as spare parts.

The tools required for replacing the parts as well as a description of the steps to take are provided with the spare part.

6.3 Unit Cover Replacement

The equipment cover (8, Fig. 1) is available as a spare part.

The spare parts are supplied with the necessary tool and installation instructions.

6.4 Fuse Replacement

Before replacing the fuse, disconnect the unit from the mains power supply by unplugging the power cord from the wall outlet.

Only replace fuses with new ones that have the same rating as those being replaced. Refer to the „Technical Specifications“ section.

Never use fuses with a higher rating

- ⇒ Unplug the unit.
- ⇒ Pull the fuse holder (17) out.
- ⇒ Remove the defective fuse and insert a new one.
- ⇒ Push the fuse holder back in, making sure it is completely seated.

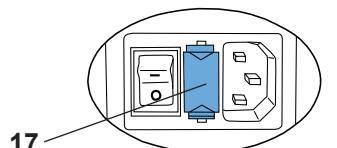
Fig. 9

6.5 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 6000 0000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

7 Troubleshooting

en

Error	Cause	Corrective action
LEDs do not light (5, 6, 7, Fig. 1), pressure plates will not move.	<ul style="list-style-type: none"> The unit is not switched on. 	<ul style="list-style-type: none"> Switch the unit on at the main power switch (16, Fig. 1).
The unit does not switch on.	<ul style="list-style-type: none"> Power cord not properly connected to the unit. Fuse blown. No power at the wall outlet. Unit fault. 	<ul style="list-style-type: none"> Check that the unit plug is properly seated (Fig. 2). Replace the fuse (refer to Sec. 6.4). Check that there is power at the wall outlet. Return the unit for repair.
Pressure plates will not advance or retract, the “equipment cover” LED (7, Fig. 1) is illuminated or is flashing.	<ul style="list-style-type: none"> Unit cover not properly closed. 	<ul style="list-style-type: none"> Close the unit cover (8, Fig. 1) and check that the cartridges are properly seated. If required, seat cartridges correctly.
Pressure plates move very slowly towards the cartridge.	<ul style="list-style-type: none"> The cover was opened before the pressure plates stopped. A previous cartridge position has been saved. 	<ul style="list-style-type: none"> Press retract key (4, Fig. 1) and wait until the pressure plates have reached their home position and have come to a complete stop. Then repeat the process with the cartridge.
Cartridges cannot be removed / cannot be installed.	<ul style="list-style-type: none"> The pressure plates have not fully retracted to their home position. 	<ul style="list-style-type: none"> Press the retract key (4, Fig. 1) and allow the pressure plates to retract until they stop automatically. They are now in their home position (refer to Chapter 5.1).
A loud sound can be heard when the mixer shaft threads itself into the mixing nozzle.	<ul style="list-style-type: none"> A spring pushes the mixer shaft forward until it reaches a stop. 	<ul style="list-style-type: none"> This is perfectly normal and requires no intervention. Only install the mixing nozzle subsequently.
The mixing nozzle (21, Fig. 1) is not turning.	<ul style="list-style-type: none"> The mixer shaft has not threaded itself. Material has hardened in the mixing nozzle. The mixer motor's thermal protector has activated. 	<ul style="list-style-type: none"> Check that the cartridge is properly seated. Install a new nozzle. Allow the unit to cool down for approx. 30 minutes, Observe the maximum permissible operating time (2 minutes of operation, 5 minute pause).
The equipment automatically reduces the extrusion speed.	<ul style="list-style-type: none"> Unit detects a highly viscous impression material and automatically reduces the feed rate and mixer rotary speed. Contamination (e.g. debris) in the cartridge holder. Insufficient spindle lubrication. Break time not maintained. 	<ul style="list-style-type: none"> This only occurs if the material is highly viscose or is cold. Refer to the manufacturer's processing instructions (processing temperature). Clean the inner surfaces of the cartridge holder and pressure plate. Lubricate the spindles (special grease!). At least 5 min. break.
The mixing nozzle bursts (21, Fig. 1).	<ul style="list-style-type: none"> Incorrect mixing nozzle installed. Mixing tip/ foil bag not compatible. 	<ul style="list-style-type: none"> Use the mixing nozzle specified by the manufacturer. Contact the manufacturer of the mixing tip / foil bag.
The tubular bag bursts or is damaged.	<ul style="list-style-type: none"> Pressure discs are too worn or have been damaged, causing the tubular bag to be pinched between the pressure disc and the cartridge. The material has partially hardened in the cartridge. 	<ul style="list-style-type: none"> Replace the pressure discs (refer to Sec. 6.2). Replace the cartridge (refer to Sec. 5.1).
Advance stops for no apparent reason and the unit performs a short relief stroke.	<ul style="list-style-type: none"> Material has hardened in the cartridge. 	<ul style="list-style-type: none"> Use new mixing nozzle.
Advance stops for no apparent reason and the pressure discs return to their home position.	<ul style="list-style-type: none"> The controller has detected an overload. Material has hardened in the cartridge. Insufficient spindle lubrication. 	<ul style="list-style-type: none"> Impression material is too cold, Refer to the manufacturer's processing instructions. Use a new cartridge. Lubricate the spindles (special grease!).

8 Technical Specifications

Item number:	6000 0000 6000 2000	6000 1000	6000 3000		
Mains voltage:	230 V	120 V	100 V		
Mains frequency:	50 Hz	60 Hz	50/60 Hz		
Power consumption:	170 VA				
Mains input fuse:	2 x T2AH	2 x T3,15 AH			
Dimensions (height x width x length):	330 x 190 x 380 mm (13 x 7,5 x 15 inch)				
Weight (empty):	7,0 kg				
Noise level:	< 70 dB(A)				
Protection class:	I				

en

9 Warranty

Provided the unit is properly used, Renfert warrants the all components of the SYMPRESS mixing unit for a period of 3 years.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer. Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee.

These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

10 Disposal Information

10.1 Disposing of Consumables

Empty cartridges and tubular bags as well as used mixing nozzles must be disposed of in accordance with the manufacturer's instructions.

10.2 Instrument Disposal

The instrument must be disposed of by a specialist facility. This specialist facility must be informed of any hazardous residue in the instrument.

10.2.1 Disposal instructions for countries in the EU

In order to maintain and protect the environment, to prevent environmental contamination and to improve the reutilisation of raw materials (recycling), the European Commission has established guidelines according to which manufacturers must take back electrical and electronic devices in order to forward them to a regulated disposal or recycling system.

Within the European Union, devices identified with this symbol may therefore not be disposed of with unsorted municipal solid waste.

For more information regarding proper disposal please apply at your local authorities.

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2 Information about electromagnetic compatibility (EMC)

2.1 General information

- The product / equipment was tested in accordance with DIN EN 60601-1-2.
- The product / equipment has no essential performance characteristics as per the definition in IEC 60601-1-2.
- The product / equipment satisfies Class A according to CISPR11:
The properties of this equipment determined by emissions allow its use in industrial applications and in hospitals (CISPR11, Class A). If use in residential applications (for which typically Class B is required according to CISPR11), the equipment may not provide adequate protection from radio services. The user may need to take remedial measures such as repositioning or realigning the equipment.
- When using the product / equipment in a **SPECIAL ENVIRONMENT**, its function may not be guaranteed nor can it be ensured that it satisfies the **ESSENTIAL PERFORMANCE CHARACTERISTICS**.
- **SPECIAL ENVIRONMENTS** may include, for example:
 - Military areas (e.g. a submarine, close to radar equipment or weapons control systems)
 - Areas of heavy industry
 - Medical treatment areas with high power medical electrical (ME) systems (e.g., RF surgical equipment, magnetic resonance imaging equipment).
- Operating in an **PROFESSIONAL HEALTH CARE FACILITY ENVIRONMENT** is permitted provided an adequate distance is kept to the high power ME systems.
- The product / equipment must not be placed and operated directly next to other electrical equipment. If this is unavoidable, the function of the product / equipment and the other electrical equipment should be checked to rule out any malfunctioning caused by electromagnetic interference.
- The use of accessories other than those provided for in the operating instructions, however they may have an adverse effect on the EMC properties.
- Stacking products / equipment is not permitted.
- Portable and mobile communication devices (e.g. cell phones, radio equipment) can affect the functioning of the product / equipment.
Maintain a distance of at least 30 cm (12 inches) between the product / equipment and mobile communication devices.

2.2 Interference Emission and Immunity Levels

2.2.1 Maintaining the Interference Emission Level

Interference Measurement	Compliance
Conducted interference (interference voltage) CISP11	Class A Limit values are complied with.
Radiated interference variable (noise field intensity) CISP11	Class A Limit values are complied with.
Distortion due to harmonics IEC 61000-3-2	Class A Limit values are complied with.
Voltage fluctuations and flicker IEC 61000-3-3	Satisfied

2.2.2 Maintaining the Interference Immunity Level

Interference Immunity Test	Standard / Test Level	Match Level
Static electricity discharge	IEC 61000-4-2 Air discharge 15 kV maximum Contact discharge: 8 kV maximum	15 kV 8 kV
High-frequency electromagnetic fields 80 MHz - 2.7 GHz	IEC 61000-4-3 10 V/m	10 V/m
High-frequency electromagnetic fields (near field) 380 MHz - 5.8 GHz	IEC 61000-4-3 28 V/m maximum	No interference
Electrical fast transients (burst)	IEC 61000-4-4 2 kV	2 kV
Surge voltages (surge)	IEC 61000-4-5 1 kV	1 kV
Conducted disturbances 150 kHz - 80 MHz / ISM + AR	IEC 61000-4-6 3 V / 6 V	3 V / 6 V
Magnetic fields with energy frequency 50 / 60 Hz	IEC 61000-4-8 30 A/m	30 A/m
Mains voltage dips and short interruptions	IEC 61000-4-11 Test voltage / exposure time 70 % / 0.5s 0 % / 0.02s 0 % / 0.01s 0 % / 5s	The response to the mains voltage dips and short interruptions is device specific. The behavior of the product / equipment in these scenarios is specified in the operating instructions for the product.

2.3 Unknown EMC Environment

Avoid operation in the vicinity of stationary transmitters such as base stations of radio telephones, land mobile radios, amateur radio stations, AM and FM broadcasting and television transmitters.

When operating in this kind of environment, pay attention to the behavior of the product / equipment.

Consider changing the location if there are any unusual functional limitations or reduced performance characteristics.

The field strength measured at the site should not exceed the above referenced compliance levels.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany