

zeta hygiene



FEEL SAFE

Hygiene solutions

Zhermack 
Dental



Zeta Hygiene,
feel safe

The background of the entire page is a dynamic, high-speed photograph of water splashing, creating numerous bubbles and droplets. The water is a clear, bright blue, and the overall effect is one of freshness and cleanliness. The splash is most prominent at the top and bottom edges of the page, framing the central text.

Cleaning, disinfection and sterilisation to protect dentists clinics and technicians laboratories against infection.

Just following the right procedure is not enough to ensure adequate protection against cross-infection between patients and personnel. Dental professionals also need **safe, effective** and **broad spectrum** products.

Rapid action, practicality and **high compatibility** with dental clinic materials make Zeta Hygiene products ideal for demanding dental professionals and guarantee a high level of safety and protection for users and patients alike.

Zeta Hygiene is a complete range of cleaning, disinfecting and sterilising products, free from phenols and aldehydes - toxic substances - and formulated for high

compatibility with clinic and laboratory surfaces. These innovative products are manufactured to the strictest quality standards and tested to ensure conformity to the European regulations.

Zeta Hygiene products deliver **effective solutions** for maintaining a **high level of protection**.

Zhermack always has your safety in mind.

Hygiene solutions



Instruments and burs

Zeta 1 Ultra page 8
Zeta 2 Sporex page 9



Surfaces

Zeta 3 Soft page 12
Zeta 3 Wipes TOTAL* page 13
Zeta 3 Foam page 14
Zeta 3 Wipes POP-UP* page 15
Zeta 4 Wash page 16

* Temporarily unavailable products



Special applications

Zeta 5 Power Act	page 18
Zeta 7 Spray	page 19
Zeta 7 Solution	page 20
Algitray	page 21
Gypstray	page 22

Hands

Zeta 6 Hydra	page 25
--------------	---------

Zhermack



dentahart

Instruments and burs



Zeta 1 Ultra



Instruments and burs

A concentrated, broad spectrum, liquid disinfectant and cleaner. Suitable for ordinary, surgical dental instruments and burs.

Active ingredients: alkylamine and quaternary ammonium salts.

2% DILUTION for a wide disinfection

1% DILUTION for a rapid disinfection

Refer to the spectrum of action indicated below.



PRODUCT BENEFITS

- ▶ High protection: broad spectrum of action.
- ▶ Effective even in the presence of organic contaminants.
- ▶ High efficiency concentrated formula.
- ▶ Safe to use: free from aldehydes and phenols.
- ▶ Easy to use: short time of action and easy dosage thanks to the dosing bottle with dispenser.

Use

2% dilution (wide disinfection): add 20 ml of product to every litre of solution. Immerse the instruments in a static bath for 60 minutes or in an ultrasonic bath for 30 minutes at 35°C.

1% dilution (rapid disinfection, with limited spectrum of action: Bactericidal, Yeasticidal, Limited virucidal): add 10 ml of **Zeta 1 Ultra** to every litre of solution. Immerse the instruments in a static bath for 15 minutes.

DEPENDING ON THE CONCENTRATION USED, EVERY LITRE OF ZETA 1 ULTRA GIVES 100 OR 50 LITRES OF DISINFECTANT SOLUTION.

Spectrum



2% DILUTION (WIDE DISINFECTION)*

STATIC BATH:

Bactericidal: EN 13727, EN 14561 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 14562 (*C. albicans*)

Bactericidal, Yeasticidal (15'): VAH

Mycobactericidal, including Tuberculocidal: EN 14348, EN 14563 (*M. terrae*, *M. avium*)

Virucidal: EN 14476 (Poliovirus, Adenovirus, Norovirus, including HIV, HBV, HCV, Ebola, Herpes simplex and all human and animal influenza viruses)

ULTRASONIC BATH:

Bactericidal, Yeasticidal, Fungicidal (*A. brasiliensis*), Mycobactericidal incl.

Tuberculocidal, Virucidal

1% DILUTION (RAPID DISINFECTION)*

STATIC BATH:

Bactericidal: EN 13727, EN 14561 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 14562 (*C. albicans*)

Limited virucidal: HIV, HBV, HCV, Ebola, Herpes simplex and all human and animal influenza viruses (DVV/RKI, prEN 16777)

Mycobactericidal including Tuberculocidal (60'): VAH

*Tests carried out in dirty conditions.

Zeta 2 Sporex



Instruments and burs

Cold chemical sterilizer and high level disinfectant in powder. Specific for dental instruments and particularly recommended for medical devices that cannot be sterilized in an autoclave.

Active ingredients: peracetic acid.

2% DILUTION



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, mycobactericidal, tuberculocidal, virucidal and sporicidal.
- ▶ Cold chemical sterilisation in just 10 minutes.
- ▶ Safe to use: free from aldehydes; peracetic acid is generated on site, avoiding the risks due to the use of free peracetic acid.

Use

2% dilution: add 20 g (3 spoons) of **Zeta 2 Sporex** to every litre of water and stir to dissolve the powder. Leave the solution to active for 15 minutes. Immerse the instruments for 10 minutes.

The presence of undissolved powder on the bottom guarantees the effectiveness of the solution for its active life according to the instructions. Once prepared, the solution remains stable for at least 24 hours, but should be replaced at the start of each working day.

ONE TUB OF ZETA 2 SPOREX GIVES 45 LITRES OF STERILISING SOLUTION.

Spectrum



Bactericidal: EN 13727, EN 14561 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 14562 (*C. albicans*)

Mycobactericidal and tuberculocidal: EN 14348, EN 14563 (*M. terrae*, *M. avium*)

Virucidal: EN 14476 (Poliovirus, Adenovirus, Norovirus, Parvovirus, including HIV, HBV, HCV)

Sporicidal: EN 13704 (*B. subtilis*)

Tests carried out in dirty conditions.



Surfaces



Zeta 3 Soft



Surfaces

Ready-to-use, alcohol-based disinfectant for the cleaning and disinfection of the surfaces of non-invasive medical devices.

Active ingredients: alcohols.

LEMON AND CLASSIC FRAGRANCES



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tuberculocidal and limited virucidal (Norovirus, Rotavirus, Adenovirus and all enveloped viruses including HIV, HBV and HCV).
- ▶ 2-in-1 product: disinfectant and cleaner in a single product.
- ▶ Compatibility tested on different surfaces.
- ▶ Reduced alcohol content, below 50%.
- ▶ Safe to use: free from aldehydes and phenols.

Use

Spray **Zeta 3 Soft** onto a disposable wipe until it is completely wet and wipe the surfaces and areas to be cleaned until all visible residues have been removed.

Repeat by spraying Zeta 3 Soft onto a new disposable wipe until it is completely wet, and wipe the previously cleaned surfaces and areas to be disinfected until they are completely wet. Wait for the 5-minute action time.

Spectrum



Bactericidal*: EN 13727, EN 16615 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 16615 (*C. albicans*)

Tuberculocidal: EN 14348 (*M. terrae*)

Limited Virucidal: EN 14476 (Norovirus, Rotavirus, Adenovirus and all enveloped viruses, including HIV, HBV and HCV)

Tests carried out in dirty conditions.

**Bactericidal according to EN 16615 in 5 minutes in clean conditions and in 10 minutes in dirty conditions.*

Zeta 3 Wipes TOTAL*



Surfaces

Alcohol solution wipes for the rapid disinfection and cleaning of the surfaces of small medical devices.

Active ingredients: alcohols.

* TEMPORARILY UNAVAILABLE PRODUCT



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tuberculocidal and virucidal vs enveloped viruses (HIV, HBV, HCV included).
- ▶ 2-in-1 action: simultaneous cleaning and disinfecting.
- ▶ Rapid drying.
- ▶ Reduced alcohol content, below 50%.
- ▶ Safe to use: free from aldehydes and phenols.

Use

Rub **Zeta 3 Wipes TOTAL** over the entire surface to be disinfected and leave to dry.

Spectrum



Bactericidal: EN 13727, EN 17387, EN 13697*, EN 14561 (S. aureus, P. aeruginosa, E. hirae)

Yeasticidal: EN 13624, EN 17387, EN 13697, EN 14562 (C. albicans)

Tuberculocidal: EN 14348, EN 14563 (M. terrae)

Virucidal vs enveloped viruses: EN 14476, EN16777 (including HIV, HBV, HCV)

Tests carried out in dirty conditions.

**Tested on S. aureus, P. aeruginosa, E. hirae, E. coli.*

Standards applicable to the solution

Effectiveness of wipes (according to prEN 16615, 10 mins contact):

R>4Log against S. aureus, E. hirae

R>3Log against C. albicans, P. aeruginosa

Zeta 3 Foam



Surfaces

A ready-to-use, alcohol-free disinfectant and cleaner foam for the cleaning and disinfection of delicate surfaces of non-invasive medical devices.

Active ingredients: 5th generation quaternary ammonium salts.

LEMON FRAGRANCE



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal and limited virucidal (Norovirus, Rotavirus, Adenovirus and all enveloped viruses, included HIV, HBV and HCV).
- ▶ 2-in-1 action: disinfectant and cleaner in a single step
- ▶ Tested compatibility with surfaces, including the most delicate ones.
- ▶ Practical: the highly visible nature of the foam makes it easy to spread over the entire surface to disinfect.
- ▶ Safe to use: free from aldehydes and phenols.

Use

Spray **Zeta 3 Foam** onto a disposable wipe until completely wet or, alternatively, spray the product directly onto the area to be treated. Wipe the surfaces and areas to be cleaned and disinfected with the disposable wipe until they are completely wet. Wait for the 5-minute action time.

Spectrum



Bactericidal: EN 13727, EN 16615 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 16615 (*C. albicans*)

Limited virucidal: EN 14476 (Adenovirus, Rotavirus, Norovirus and all enveloped viruses, including HIV, HBV, HCV)

Tests carried out in dirty conditions.

Zeta 3 Wipes POP-UP*



Surfaces

Large wipes impregnated with a weak alcohol solution for rapid disinfection and cleaning of the surfaces of even the most delicate medical devices.

Active ingredients: quaternary ammonium salts.

* TEMPORARILY UNAVAILABLE PRODUCT



PRODUCT BENEFITS

- ▶ High protection: bactericidal, fungicidal, yeasticidal, tuberculocidal and virucidal.
- ▶ Strong, soft and compact fabric.
- ▶ Safe to use: free from aldehydes and phenols.

Use

Rub **Zeta 3 Wipes POP-UP** thoroughly over the entire surface to be cleaned, making sure that the surface is left evenly damp and leave the product work for.

Spectrum



Bactericidal: EN 13727*, EN 13697, EN 1276, EN 14561*, EN 14561 (MRSA)

Fungicidal: EN 14562 (A. fumigatus)

Yeasticidal: EN 1650*, EN 13624, EN 13697, EN 14562 (C. albicans)

Tuberculocidal: EN 14348, EN 14563 (M. terrae)

Virucidal: EN 14476* (HBV, HCV, Adenovirus, Coronavirus, Norovirus, VRS, H1N1, HSV1), EN 14476 (Polyomavirus). EN 14476 (Rotavirus)

Tests carried out in clean conditions.

** Tests carried out in dirty conditions.*

Zeta 4 Wash



Surfaces

A concentrated cleaner and deodorising solution for washable surfaces.

Active ingredients: non-ionic and cationic surfactants.



PRODUCT BENEFITS

- ▶ Ensures rapid, deep cleaning and leaves a fresh, clean fragrance.
- ▶ Compatible with all materials.
- ▶ Over 90% biodegradable.



Use

Add 1-2 dosing caps to every litre of water. For cleaning ingrained dirt, use a few drops of the concentrated solution on a damp cloth.

Special applications



Zeta 5 Power Act



Special applications

A specific concentrated disinfectant and cleaner for suction units and spittoons.

Active ingredients: alkylamine and quaternary ammonium salts.

1% DILUTION



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tuberculocidal and limited virucidal.
- ▶ Rapid acting formula: disinfects in 15 minutes.
- ▶ Tested compatibility with the suction unit components.
- ▶ Non-foaming and non-aggressive to vacuum tubes.
- ▶ Safe to use: free from aldehydes and phenols.
- ▶ Easy to use: short time of action and easy dosage bottle with dispenser



Use

1% dilution: add 10 ml of **Zeta 5 Power Act** to every litre of water and stir to dissolve. Prepare at least one litre of solution per dental unit. Aspirate the prepared solution through the vacuum system. Leave the solution work for 15 minutes to achieve rapid disinfection (excluding tuberculocidal action) or leave overnight for complete action. Pour at least 250 - 300 ml of disinfectant solution into the rinsing basin and leave to work.

ONE LITRE OF ZETA 5 POWER ACT GIVES 100 LITRES OF DISINFECTANT SOLUTION.

Spectrum



Bactericidal: EN 13727, EN 14561 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624, EN 14562 (*C. albicans*)

Tuberculocidal: EN 14348 (*M. terrae*)

Limited virucidal: prEN 16777, DWV/RKI against encapsulated and lipophilic viruses including haematic viruses (HIV, HBV, HCV), herpes simplex and virus families such as orthomyxoviridae (including all forms of human and animal influenza virus such as H5N1 and H1N1), filoviridae (ebola virus) and paramyoviridae (measles virus)

Bactericidal, Yeasticidal: VAH

Tests carried out in dirty conditions.

Zeta 7 Spray



Special applications

A ready-to-use disinfectant spray for the rapid disinfection of impressions.

Active ingredients: alcohols.



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tuberculocidal and virucidal.
- ▶ Compatibility with materials for the impression taking (addition and condensation silicones, alginate, polyether, polysulphide and polyvinyl).
- ▶ Safe to use: free from aldehydes and phenols.
- ▶ Practical, fast and ready to use, without rinsing, disinfects up to 250 impressions.

Use

Rinse the impression under running water. Spray **Zeta 7 Spray** onto both sides of the impression to obtain a continuous film of disinfectant covering the two surfaces. Leave to dry for at least 3 minutes.

RESPECTS THE DIMENSIONAL STABILITY OF IMPRESSIONS.

Spectrum



Bactericidal: EN 13727 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624 (*C. albicans*)

Tuberculocidal: EN 14348, EN 14563 (*M. terrae*)

Virucidal: EN 14476 (tested on Poliovirus, Adenovirus, Norovirus, Parvovirus, including HIV, HBV, HCV)

Tests carried out in dirty conditions.

Zeta 7 Solution



Special applications

A concentrated disinfectant for impressions.

Active ingredients: quaternary ammonium salts, phenoxyethanol.

1% DILUTION



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tuberculocidal and virucidal against enveloped viruses.
- ▶ Compatibility with impression materials (addition and condensation silicones, alginate, polyether, polysulphide and polyvinyl).
- ▶ High efficiency concentrated formula.
- ▶ Respects the dimensional stability of impressions and their compatibility with gypsum.
- ▶ Safe to use: free from aldehydes and phenols.

Use

1% dilution: add 10 ml of **Zeta 7 Solution** to every litre of water. Rinse the impression under running water. Immerse the impression in the disinfectant solution for 10 minutes. Remove the impression from the bath and rinse it thoroughly.

ONE LITRE OF ZETA 7 SOLUTION GIVES 100 LITRES OF DISINFECTANT SOLUTION. (UP TO 40 IMPRESSIONS CAN BE DISINFECTED WITH ONE LITER OF SOLUTION*).

*The solution must be replaced every day and immediately in case of visible contamination (blood, saliva or organic tissue).

Spectrum



Bactericidal: EN 13727 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

Yeasticidal: EN 13624 (*C. albicans*)

Bactericidal, Yeasticidal: VAH (30')

Tuberculocidal: EN 14348 (*M. terrae*)

Virucidal against enveloped viruses: EN 14476 (Vaccinia Virus including HIV, HBV, HCV)

Tests carried out in dirty conditions.

Algitray



Special applications

A specific cleaner, free of phosphates and surfactants, for removing alginate residues from impression trays and other instruments.



PRODUCT BENEFITS

- ▶ Non-hazardous for the environment and for users: contains up to 98% of biodegradable detergent ingredients.
- ▶ High performance: quickly removes all traces of alginate from impression trays and other instruments.
- ▶ Safe to use on instruments: non-aggressive to metals thanks to a neutral pH.

Use

Add 100 g of **Algitray** (4 spoons) to 1 litre of water and stir gently until completely dissolved. Remove as much alginate from the impression trays or instruments as possible. Immerse the impression trays or instruments in the solution.

Leave to react for about 15-30 minutes. Remove the impression trays or instruments from the solution. Brush off all remaining traces of alginate then rinse under running water.

1 KG TUB OF ALGITRAY GIVES 10 LITRES OF CLEANING SOLUTION.



Gypstray



Special applications

A ready-to-use solution for removing residues of gypsum and gypsum coatings from impression trays, spatulas or other instruments.



PRODUCT BENEFITS

- ▶ High performance: quickly removes all traces of gypsum from impression trays and other instruments.
- ▶ Practical and ready to use.
- ▶ Safe on materials: a non-aggressive formula guarantees safe use every time.


Use

Pour out enough **Gypstray** to completely cover the items to clean. Immerse the instruments or trays and leave the solution to work for about 15-30 minutes, depending on the type and quantity of residues. Effervescence indicates that the solution is working.

Remove the items from the solution and rinse with water.



Hands

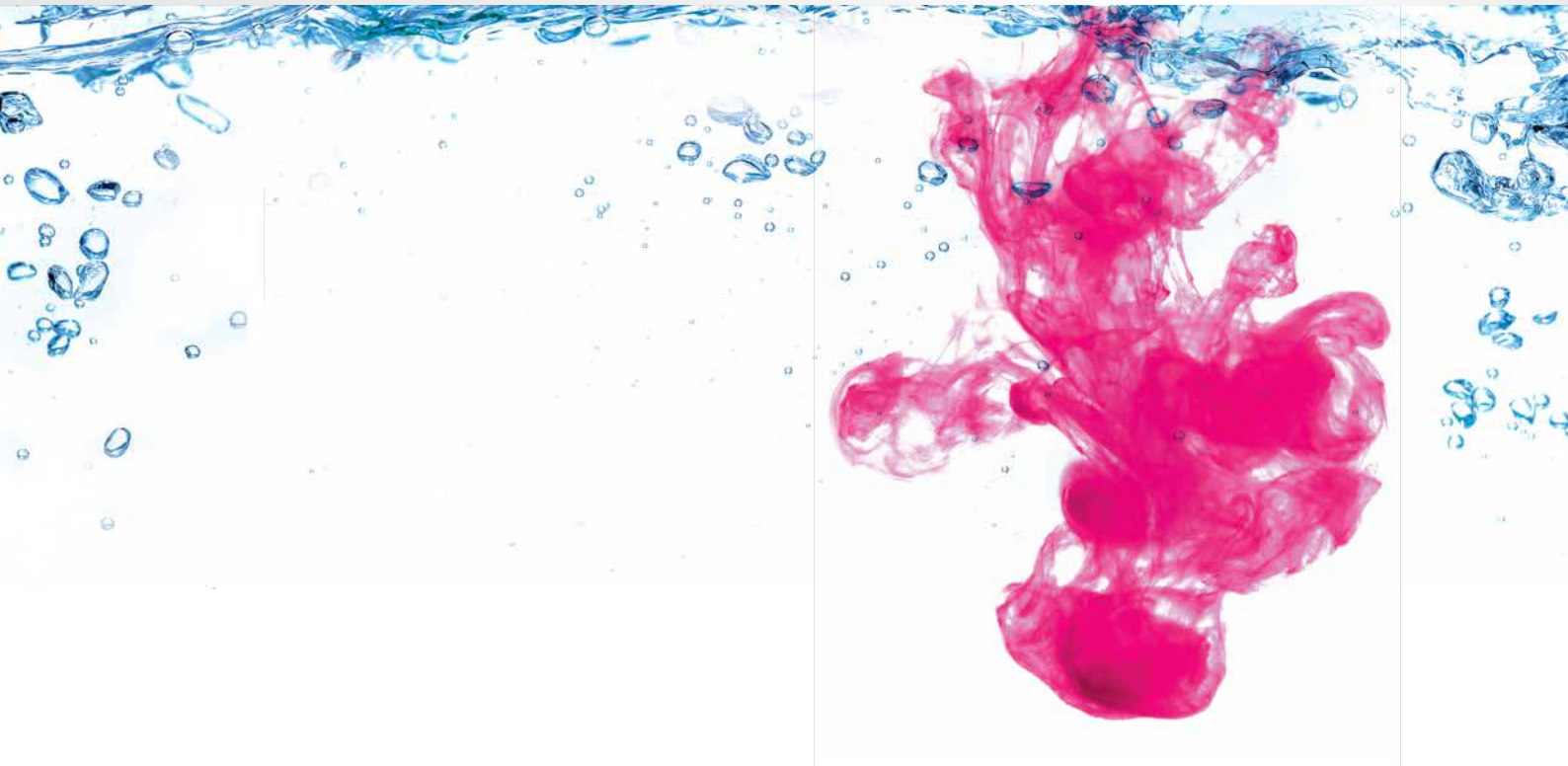



Zeta 6 Hydra



Hands

A delicate cleaner for frequent use on hands and delicate skin.

Active ingredients: aqua (water), cocamidopropyl betaine, sodium cocoamphoacetate, lauryl glucoside, glycerin, ammonium lauryl sulfate, glyceryl oleate, hydrolyzed rice protein, coco glucoside, sodium lauryl glucose carboxylate, sodium cocoyl glutamate, dehydroacetic acid, benzyl alcohol, citric acid, disodium EDTA, parfum (fragrance).



PRODUCT BENEFITS

- ▶ Nourishing, protective and moisturising action.
- ▶ Also suitable for washing face and body.

Use

Wet the hands and pour **Zeta 6 Hydra** into the palm of one hand. Rub the hands together thoroughly then rinse and dry.

Zeta 6 Hydra contains a mix of vegetable substances with the following properties:

- nourishing, thanks to hydrolyzed rice protein, which restores the skin's natural softness
- protective, thanks to glyceryl oleate, which preserves and helps restore the lipidic structure of the skin's outer layers
- moisturising, thanks to glycerin, which helps prevent dryness and irritation

Packaging



Disinfectants and sterilising agents for ordinary, surgical dental instruments and burs

Code	Product	Packaging
C810000	Zeta 1 Ultra	1 litre bottle
C810011	Zeta 2 Sporex	900 g tub with dosing spoon



Disinfectants and cleansers for the surfaces of medical devices

Code	Product	Packaging
C810023	Zeta 3 Soft	750 ml bottle with spray cap
C810024	Zeta 3 Soft	5 litre canister (2 x 2.5 litres) with dispenser
C810027	Zeta 3 Soft Classic	750 ml bottle with spray cap
C810028	Zeta 3 Soft Classic	5 litre canister (2 x 2.5 litres) with dispenser
C810025	Zeta 3 Foam	750 ml bottle with foam dispenser
C810026	Zeta 3 Foam	3 litre canister with dispenser
C810037	Zeta 4 Wash	3 litre canister





Special applications Disinfectants and cleaners for suction units

Code	Product	Packaging
C810040	Zeta 5 Power Act	1 litre bottle



Special applications Disinfectants for impressions

Code	Product	Packaging
C810050	Zeta 7 Spray	750 ml bottle with spray cap
C810048	Zeta 7 Solution	1 litre bottle



Special applications Cleaners for cleaning and removing alginate and gypsum residues from instruments

Code	Product	Packaging
C400435	Algitray	1 kg tub with dosing spoon
C400441	Gypstray	3 litre canister



Cleaner for hand hygiene

Code	Product	Packaging
C810042	Zeta 6 Hydra	1 litre bottle with dosing cap
C810043	Zeta 6 Hydra	5 litre canister with dispenser

Fulfilling your needs