


ELITE TRANSPARENT

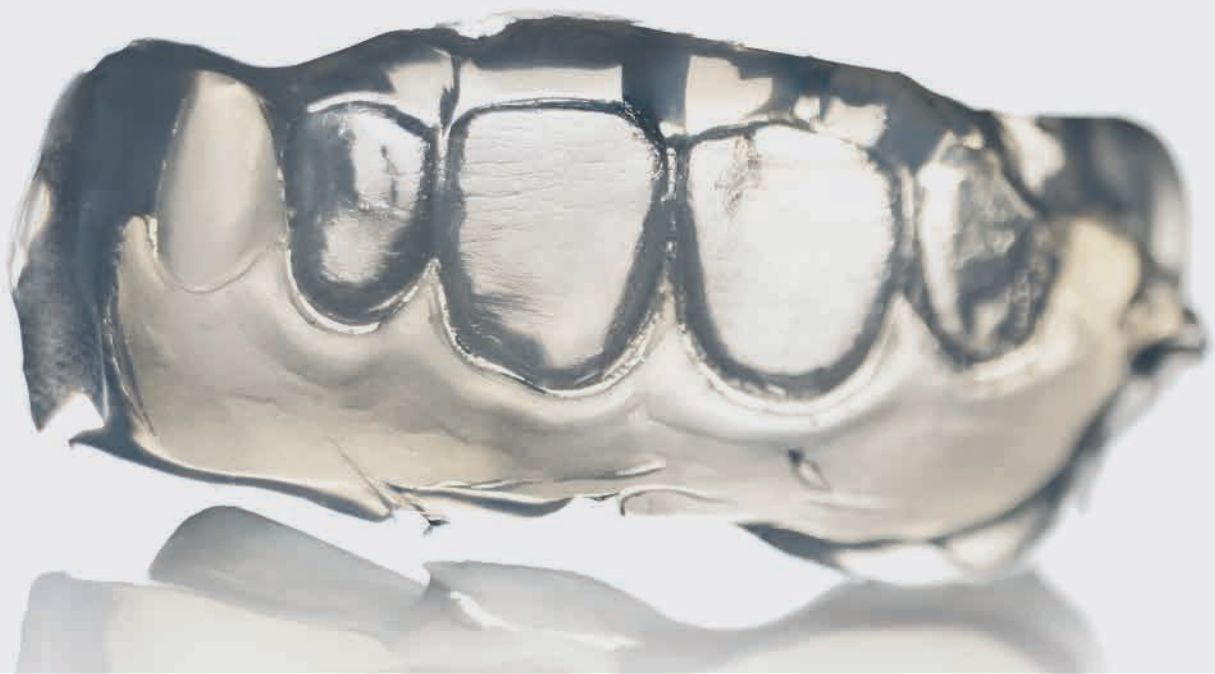

USER'S GUIDE

A-Silicone for masks for light-curing materials

Elite Transparent, simplify your work

**Elite Transparent is a transparent fluid
two-component addition silicone.**

Formulated for making transparent silicone masks
in the laboratory for composites and light-curing
resins.



1.

TEMPORARY RESTORATIONS WITH ELITE TRANSPARENT

The combined use of transparent silicones and light-curing composites for the preparation of aesthetic temporary restorations saves plenty of time without foregoing quality.

Materials used: Elite Transparent, Elite Rock.

Model



Wax-up



Application of Elite Transparent





Application of a transparent film to smooth the mask. Once this phase is complete, during silicone curing, the model can be placed under pressure at 2 bars



Cut back (dentine reduction to make space for the enamel)



9

Filling the mask with composite (enamel)



10

>>

Mask repositioning
and light cure



11

FINAL RESULT
after finishing and polishing





Technical features

Working time* (min:s)	Setting time* (min:s)	Linear dimensional change (after 24 h)	Hardness (Shore A after 1 h)	Tensile strength	Tear strength
1:30	15:00	0.18 %	72	6.5 N/mm ²	4 N/mm

*The times mentioned above are intended from the start of the mixing phase at 23° C (73° F).

Codes



Elite Transparent - A-Silicone for masks

Code	Packaging
C401600	1 x 50 ml cartridge + 6 mixing tips (small)

Find out more about related Zhermack products for masks for light-curing materials



Elite Rock

Extra-hard type 4 stone for master models in fixed prosthesis applications

For more information please visit our website: www.zhermack.com



Fulfilling your needs