IT - Elite LC Tray

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EN - Elite LC Tray

Instructions for use
Patient target group <p>All persons for whom individual trays are required for treatment.</p>
Intended user <p>Dentist</p>

Compositon Qualitative data:

Oligomere urethane acrylate, vinyl ester resin, 2,2-dimethoxy-1,2-diphenylethane-1-one or by 2,2-dimethoxy-2-phenylacetophenone, (+)-2,3-dioxo-1,7,7-trimethyl-bicyclo-(2,2,1)-heptane di-bornano-2,3 diione, difenil(2,4,6-trimetilbenzoli)fosfinaossido, 2-idrossio-2-metil-1-fenil-propa-1-ono, 2-etilheks-4-(dimetilammino)benzoato, 2,2 Dietiosiacetofenone, vetro sodocalico, silice amorfa, coloranti (pigmento rojo 146 = CI 12485 / pigmento azul 29 = CI 77007 / D&C rojo 30 = CI 73360 / pigmento azul 19 = CI 73900), aromas (Optamint).

Quantitative data:

In 100 g hay 19 g acrylates/metacrylates and 80 g of glass.

Intended purpose of the product

Elite LC Tray / Elite LC Tray Round is a light-curing tray material in the form of prefabricated, basplesets to make rigid, custom impression trays and bite-blocks, as well as for other indications. Elite LC Tray / Elite LC Tray Round can be cured in all ultraviolet and Halogen light-curing units. The basplesets are supplied in a re-useable light-proof box that can be used for transporting or storing models, etc.

Indications

Custom impression tray

Contraindications

The material contains methacrylates.

Do not use in patients with an allergy to methacrylates or other substances contained in the product.

Performance characteristics of the product

Elite LC Tray / Elite LC Tray Round polymerises with light to form a solid and compact polymer.

Specific precautions
Avoid any contact between the eyes and skin and the uncured material, i.e. also with the inhibited layer. Always use a dust extractor when trimming the tray!

Preparation/Manufacture

- Mark the outline of the future tray on the model.

- Block out the undercuts on the model.

- To create a spacer for the impression material, block out the entire model (e.g. with a sheet of wax) and out of stops.

- If the plaster is very dry, soak the model briefly in water or apply dry isolation or an alginate isolation.

- Remove the model from the box and close it.

- Once the model has been blocked out, adapt the basplate to it and cut to shape using, e.g. a scalpel.
- Use the remaining material to form a handle and attach it to the tray.

- Place the model and tray in a light-curing unit and cure without vacuum.

- Remove the tray from the model and polymerize once again from the underside.

- If the tray has been cured without a vacuum, use alcohol to remove the smear layer.

Working at daylight	20 minutes
Polymerisation in an ultraviolet or Halogen light-curing unit, depending on the unit in use:	
The top of the tray, on the model	3-5 minutes
The underside of the tray, off the model	3-5 minutes
Trimming:	
The custom tray with tungsten carbide burs and smooth it with sandpaper.	

Immense impraesiones taken with Elite LC Tray / Elite LC Tray Round custom trays in a disinfectant solution based on glutaraldehyde. The time depends on the disinfectant manufacturer's directions. The finished medical product is used by the dentist, e.g. for individual trays with impression material.

Caution

The product has been developed for use in the dental sector and must be used according to the di-rections. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore the user is obliged under his own responsibility to examine the material for its suitability and its usage possibilities for the intended applications, especially if the latter are not detailed in the directions for use.

Side effects

Adverse side effects are not known. Immerse impraesiones taken with Elite LC Tray / Elite LC Tray Round custom trays in a disinfectant solution based on glutaraldehyde. The time depends on the disinfectant manufacturer's directions. The finished medical product is used by the dentist, e.g. for individual trays with impression material.

Interactions

Interaction of the product and its metcraltyes with other medical devices, medicinal products or other substances is not known.

Technical data

Flexural strength min. 130 N/mm²

Minimum shelf life

Do not use Elite LC Tray / Elite LC Tray Round after the expiry date.

Variants

- Elite LC Tray **[REF: D500100 White]** **[REF: D500101 Pink]** **[REF: D500102 Blue]**
- Elite LC Tray Round **[REF: D500052 Pink]**

Disposal

Uncured material can be polymerised with light (sunlight or artificial light) and then disposed of as plastic waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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