USER MANUAL

Alginate Mixer

MX-300

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Alginate Mixer MX-300

Table of contents:

- 1. Preface**
- 2. Note**
- 3. Important subjects of accident prevention and malfunction**
- 4. Name of each portion**
- 5. Installation**
- 6. Operation method**
- 7. Setting for mixing time**
- 8. Maintenance**
- 9. Trouble shooting**
- 10. Specification**
- 11. Introduction**
- 12. General cleaning procedures**
- 13. Fuse replacement information**
- 14. Electromagnetic interference**
- 15. Waste disposal**
- 16. Warranty**
- 17. Spare parts list**

1. Preface

Thanks for your purchasing MX-300 Alginate Mixer. Prior to using the product, please read this USER MANUAL carefully and its correct maintenance to enable the function can be fully implemented. After reading this USER MANUAL, please keep it properly. If there is any further improvement or the contents of this manual shall have any further amendment, no notice shall be otherwise advised.

2. Note

- 2.1 We shall not assume any responsibility if any malfunction, damage, or accident of personal body injure is caused by the improper removal, modification, maintenance, or repair as not being conducted by the dealer designated by us.
- 2.2 We shall not assume any responsibility if any malfunction, damage, or accident of personal body injure is caused by the use of the product not provided by us.
- 2.3 We shall not assume any responsibility if any malfunction, damage, or accident of personal body injure is caused by the improper removal, modification, maintenance, or repair as not use the parts provided by us.
- 2.4 We shall not assume any responsibility if any malfunction, damage, or accident of personal body injure is caused by the improper operation as not in compliance with the manual.
- 2.5 We shall not assume any responsibility if any malfunction, damage, or accident of personal body injure is caused by the ambient environment as not in compliance with the power, installation environment and using conditions specified on this manual.
- 2.6 We shall not assume any responsibility if any malfunc-

tion, damage, or accident of personal body injure is caused by fire disaster, earthquake, flood, or struck by lightning.

3. Important subjects of accident prevention or malfunction

Prior to reading the operation of "ALGINATE MX-300", please read first the "Important notes" marked. The operation shall be in compliance with "Important Notes" and "Notes for the Use of Medical Therapeutic Machine (Safety and Danger Prevention)" specified on the manual so as to avoid any malfunction in advance, and please use correctly to avoid any personal injure.

Important notes for the possible personal injure while in use

The operation shall be in compliance with "Important Notes" and "Notes for the Use of Medical Therapeutic Machine (Safety and Danger Prevention)" specified on the manual so as to avoid any malfunction in advance, and please use correctly to avoid any personal injure.

⚠ Danger

It is possible to cause death or serious injure.

⚠ Warning

It may cause malfunction or damage to the machine.

⚠ Notes

It is possible to cause slight personal injuries or damage to machine.

⚠ Important

It should be noted closely, or possible to cause malfunction of machine.

Important information

The responsibility for the operation of medical therapeutic machine and maintenance inspection is attributable to the user (hospital or clinic).

The operation of medical therapeutic machine shall be conducted by the authorized person (assistant, dentist, and nurse).

The purpose of this device is to automatic mix alginate powder and water in order to take dental impressions.

Alginate mixer MX-300 reduces air bubbles inside the mixture in order to reach a better result in the final impression.

⚠ Notes

Any serious incident that has occurred in relation to this device should be reported to Motion Dental Equipment Corporation, Our EU representative, and related government authorities.

Our EU representative information is located on the label sticker on the mixer.

Models		Standard delivery	
C305170	MX-300; 230 V; 50/60 Hz	1 MX-300 mixing unit 1 Manual 1 Power cord 1 Spatula	4 Mixing bowl 1 Stirring stick 1 Water dosing bottle 1 Magnetic basket
C305171	MX-300; 115 V; 50/60 Hz		
C305172	MX-300 for UK; 230 V, 50/60 Hz		

Spare parts: Refer to the enclosed spare parts list for the order number of spare parts

4. Name of each portion (Fig. A-B-C)

- | | | | |
|--------------------------|-----------------|-----------------|-------------------------|
| 1. Main power switch | 5. Up button | 9. Fuse holder | 13. Stirring stick |
| 2. Power switch (on/off) | 6. Down button | 10. Mixing bowl | 14. Water dosing bottle |
| 3. Start/Stop button | 7. Power cord | 11. Cover body | 15. Magnetic basket |
| 4. Timer display | 8. Power socket | 12. Spatula | 16. Motor screw |

5. Installation

5.1 Prior to installation please remove the motor screws fixed at the bottom of the machine. If the device needs to be transported, reinstall the motor screws to secure the motor for delivery (Fig. 1).

5.2 After removing the motor screws from the bottom of the machine, fix them in the appropriate holes (Fig. 14).

6. Operation method

- 6.1 Turn on the main power switch of the alginate mixer located on the backside of the machine (Fig. 2 and Fig. 3).
- 6.2 Press the power button located on the control panel. The LED digital display on the operation panel will now be lit (Fig. 4 and Fig. 5).
- 6.3 Press the "UP" or "DOWN" arrows to adjust and set your preferred mixing duration in seconds.
- 6.4 Prepare the alginate powder and water (10 °C - 20 °C) (Fig. 6). Note: Alginate setting time can be affected by the water temperature.
- 6.5 Apply suggested ratio of alginate powder to water recommended by the supplier. The amount of water added will directly affect the viscosity of the mixture.
- 6.6 If there is an excessive amount of alginate powder, slightly stir it by using the stirring stick (Fig. 7). Important: make sure the cap is fully tightened before mixing! Failure to do so could result in device malfunction or damage. If the cap cannot be fully tightened due to damage, please replace with a new cap.
- 6.7 Lock the cap by turning it clockwise (Fig. 8 and Fig. 9).
- 6.8 Place the mixing bowl into the mixing well (Fig. 10).
- 6.9 Close the top cover and make sure the top cover is fully closed before starting to mix.
- 6.10 Push the "START/STOP" button to start the mixing process. The duration will remain programmed until manually changed (Fig. 11 and Fig. 12).

7. Setting for mixing time

7.1 The table below shows the recommended mixing time for Zhermack brand alginates.

Alginate	Mixing Duration (sec)
Hydrocolor 5	6
Hydrogum 5	6
Hydrogum	10
Orthoprint	6
Tropicalgin / Phase	8
Neocolloid	10
Hydrogum Soft	8
Pinkalgin 5	6

7.2 MX-300 accelerates the working time, initial setting time and setting time stated in the manufacturer instructions of the alginate; use cold water to limit the phenomenon.

7.3 Manufacturer timer settings can be altered for desired mixing times. Set the timer with the up and down arrows to the desired time. Then press and hold both the "UP" and "DOWN" arrow keys simultaneously for 2 seconds until an audible beep occurs. The mixing time display panel will flash twice. Up to 3 different memory presets can be made.

7.4 The following are general guidelines for not Zhermack brand alginates mixing times with water between 10 °C - 20 °C

Spoons	Mixing Time (second)
	8
	10
	12

8. Maintenance

8.1 Make sure to disinfect, and if possible sterilize (non-autoclavable material, cold sterilize only), all accessories used together with the alginate before and after use. Follow warnings concerning material compatibility, maintenance and refurbishment provided by the accessory manufacturer. Follow the chosen disinfectant manufacturer's instructions for contact time. Please refer to the instructions for use for each specific disinfectant you wish to use to check compatibility and effectiveness.

8.2 Mixing bowl: After setting, the remaining impression material inside the bowl can be easily remove with a spatula and gently clean with water or with non-alcohol based products. The small hole at the center of the cap should be cleaned as well (Fig. 13).

8.3 If there is any residue or debris on the mixer, use a dry cloth to wipe down the unit. DO NOT use a solvent or volatile oil to clean the unit as this could cause surface discoloring.

9. Trouble shooting

Scenario	Cause	Solution
No power	Bad connection of power cord	Check power connection
	Burned-out fuse	Inspect fuse and replace if needed (there is a spare one inside the holder)
Irresponsive digital display	Defective PC board	Replace the whole part
	Defective motor or gear	Replace the whole part
	Top cover not fully closed	Make sure the top cover is fully closed
Slow start up	Defective motor or gears	Replace the whole part
	Defective starting capacitor	Replace the whole part
Unusual noise or vibration during operation	Too much alginate powder or water	Always apply less than 100 grams of alginate powder before mixing
	Empty mixing bowl	Never mix with an empty mixing bowl
	Damaged gears	Replace the whole part
	Alginate powder spill	Fully tighten the cap

10. Specification

Dimensions	W205 x D245x H300 (mm)
Weight	17 Kg
Voltage	AC220 V - AC240 V (50/60 Hz) / AC100 V - AC120 V (50/60 Hz)
Maximum Power	400W
Timer	1-20 sec, adjustable by time
Frequency	2,990 rpm (50Hz) / 3,600 rpm (60Hz)

11. Introduction

Purpose of use: This product is for mixing alginate powder

Applied part: None

Classification: Class I

AP, APG: None

Emergency switching of disconnected device: By removing the power cord plug

Storage temperature: -20 °C to 60 °C (-4 °F ~140 °F)

Storage/transportation humidity: 35-95% RH

Storage atmospheric pressure: 700~1060hPa

Operation temperature: 5 °C - 40 °C

Operation time: Max. 20 seconds for each use

Rest period: 1 minutes after each use

12. General cleaning procedures

⚠ Caution: Prevent leakage of fluids onto the unit and see "safety precaution" sheet inside the box.

12.1 Cleaning (Gently wipe it with a piece of soft dry cloth)

12.2 Cleaning (Gently wipe it with a piece of cloth moistened with water)

13. Fuse replacement information

⚠ Warning

Electric shock hazard: During fuse replacement, make sure that all power to the system is shut off.

14. Electromagnetic interference

⚠ Caution: Keep this machine away from other equipment that will generate electromagnetic or other interference.

The device has been tested to meet the following standards:

IEC/EN60601-1, IEC/EN60601-1-2, MDR 2017/745, and ISO13485

15. Waste disposal

⚠ Warning

Disposal hazard: This product contains materials which may require disposal through appropriately licensed hazardous waste management.

The following materials are contained in the Alginate Mixer. Ensure proper disposal of hazardous and other regulated waste in compliance with national and local regulations.

- Metal parts: Mainly made from steel, iron alloys, aluminum, aluminum alloys, copper, and copper alloys.
- Electrical parts: Mainly PC boards and electronic components.
- Other Materials: Mainly plastic, ceramic, silicon, and synthetic rubber.

16. Warranty

MX-300 is covered by a warranty of one year from the date of purchase. Warranty claims may only be made upon presentation of the original sales receipt/invoice from the authorized dealer. Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list. The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed or in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

In the event of replacement of the unit the period of warranty of one year doesn't change and remains connected to the first unit (serial number) mentioned to the original sales receipt/invoice from the authorized dealer.

IMPORTANT: Always keep the original product packaging throughout the guarantee period. You will need the packaging if the product has to be returned to the Service Centre for repairs or replacement in the event of manufacturing defects or malfunction (with the exception of willful damage, negligence, failure to perform routine maintenance or damage during transport). The customer is responsible for all costs associated with unsuitable packaging of the product to be returned to the Service Center. Motion is not liable for faults, malfunctions or damage to the product deriving from or consequent to the use of unsuitable packaging, or packaging not of a quality equal to or better than the packaging used for the original shipment.

17. Spare parts list

Item code	Note	Description
XR0251230	(wear parts)	Mixing bowl
XR0251231	(wear parts)	Spatula
XR0251232	(wear parts)	Stirring stick
XR0251233	(wear parts)	Magnetic basket
XR0251234	(wear parts)	Water dosing bottle
XR0251235	(wear parts)	Gear system (gear axle)
XR0251236	(wear parts)	Gear system (gear wheel)
XR0251237		Bearing (pair)
XR0251238		Gear system (Big gear wheel)
XR0251239		Power board
XR0251240		Display board
XR0251241		Magnetic safety lock (230 V)
XR0251242		Magnetic safety lock (115 V)
XR0251243		Power motor (230 V)

Item code	Note	Description
XR0251244		Power motor (115 V)
XR0251245		Dust-proof cover
XR0251246		Metal mixing bowl holder
XR0251247		Brake set (230 V)
XR0251248		Brake set (115 V)
XR0251249		Top cover
XR0251250		Middle case
XR0251251		Lower case
XR0251252		Control panel
XR0251253		Rubber ring
XR0251254		Power cord EU plug (230 V)
XR0251255		Power cord US plug (115 V)
XR0251256		Power cord UK plug (230 V)
XR0251257		Hinges (pair)

MD This symbol identifies a medical device as defined by EU Regulation 2017/745.

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Zhermack S.p.A.