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RANGE OF ACTION	Zeta 3 Foam	
	This product has been designed for use by qualified operators with knowledge of the required safety standards. Use protective gloves and appropriate PPE. Operate in well-ventilated areas. Store the closed container in a cool dry place. Avoid contact with aluminium, brass and unalloyed iron surfaces. Prevent Zeta 3 Foam from coming in contact with other disinfectants or cleaners, particularly with anionic or aldehyde surfactants (formaldehyde, glutaraldehyde, glyoxal, etc.). Any slight difference in the perfume is due to the presence of natural fragrance and does not jeopardise quality of the product. Once the container has been opened and the applicator inserted, keep in a vertical position.	
Efficacy (microorganisms)	Reference standard <i>Tests carried out in dirty conditions.</i>	Reaction Time (minutes)
Bactericidal (S. aureus, P. aeruginosa, E. hirae)	EN 13727, EN 16615*, EN 13697	5
Yeasticidal (C. albicans)	EN 13624, EN 16615, EN 13697	5
Tuberculocidal (M. terrae)	EN 14348	5
Limited Virucidal (Norovirus, Rotavirus, Adenovirus and all enveloped viruses, included HIV, HBV and HCV)	EN 14476	5

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EN - Alcohol-free foam spray disinfectant and cleaner for delicate invasive and not invasive medical device surfaces. Zeta 3 Foam is a disinfectant and cleaner free from aldehydes, ready-to-use and with a broad spectrum of action. Its excellent compatibility with materials makes Zeta 3 Foam particularly suitable for the disinfection between patients of delicate medical device surfaces (handpieces, contra-angles, dental units, etc.).

Compositio: 100 g of Zeta 3 Foam contains 0.315 g dimethyl-dicetyl-ammonium chloride, 0.075 g alkylbenzyl-dimethyl ammoni-um chloride, non-ionic surfactants, additives, auxiliaries and water to 100 g.

Range of action: Bactericidal (S. aureus, P. aeruginosa, E. hirae and E. coli only in compliance with EN 16615), Yeasticidal (C. albicans), Tuberculocidal (M. terrae) in 5 minutes and limited Virucidal (Adenovirus, Rotavirus, Norovirus and all enveloped viruses, including HIV, HBV and HCV) in 5 minutes. Activity tested in compliance with EN13727, EN16615, EN13697, EN13624 and EN14348. L'activity virucida is tested in accordance with EN14476. For further details on reaction time and efficacy, please refer to the table.

Use: Dispense Zeta 3 Foam and use it to spread it so that it completely covers the surfaces and objects to be disinfected (4 sprays of Zeta 3 Foam are sufficient to cover a surface area of 0.25 m²). Leave it to work, then remove the disinfecting emulsion with a tissue and finally leave to dry. Alternatively, it is possible to spray Zeta 3 Foam on a tissue and wipe it over surfaces and areas to be disinfected. It is recommended to follow the times indicated in the table. Exposures for longer times could lead to a long-term deterioration of the treated surfaces.

Warnings, precautions and conservation conditions

This product has been designed for use by qualified operators with knowledge of the required safety standards. Use protective gloves and appropriate PPE. Operate in well-ventilated areas. Store the closed container in a cool dry place. Avoid contact with aluminium, brass and unalloyed iron surfaces. Prevent Zeta 3 Foam from coming in contact with other disinfectants or cleaners, particularly with anionic or aldehyde surfactants (formaldehyde, glutaraldehyde, glyoxal, etc.). Any slight difference in the perfume is due to the presence of natural fragrance and does not jeopardise quality of the product. Once the container has been opened and the applicator inserted, keep in a vertical position.

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