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SOLO PER USO PROFESSIONALE

EN - INSTRUCTIONS FOR USE: Disinfectant and cleaner for medical device surfaces.

PACKS: 2500 ml container.

ACTIVE INGREDIENTS: 100 g of Zeta 3 Soft and Soft Classic contain 34.4 g ethanol, 14 g isopropanol, non-ionic surfactants, additives, auxiliaries and water to 100 g.

CONTRAINDICATIONS: No known contraindications.

WARNINGS: We always recommend using the product on a small area before proceeding to treat the entire surface. Flammable liquid and vapors. Causes serious eye irritation. In the event of contact with eyes, rinse thoroughly for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If the eye irritation persists, consult a doctor.

Contains: (R)-P-MENTHA-1,8-DIEN. May cause an allergic reaction. If irritation, reddening or other signs of hypersensitivity occur during use, stop the eyes and consult a doctor.

PRECAUTIONS: Wear gloves and protect using and top. Wash the hands thoroughly after use. Keep away from sources of heat, sparks, naked flames and hot surfaces. Do not smoke. Keep in a tightly-closed container. Do not dispose of the product in the environment; dispose in accordance with local laws and regulations.

SIDE EFFECTS: No side effects have been noted.

STEP-BY-STEP INSTRUCTIONS: Spray the product onto the surface to be disinfected directly or through a cloth. Wipe the entire surface to be disinfected thoroughly before the product has evaporated completely, then leave to dry.

SPECTRUM OF ACTION

Activity	Standards*	Time (minutes)
Virucidal	EN 14476 (tested on Poliovirus, Adenovirus, Norovirus, Parvovirus, including HIV, HBV, HCV, H1N1)	1
Bactericidal	EN 13727, EN 14561 (S. aureus, P. aeruginosa, E. hirae)	5
Yeasticidal	EN 13624, EN 14562 (C. albicans)	5
Tuberculocidal	EN 14348, EN 14563 (M. terrae)	5

*Tests performed in dirty conditions.

IMPORTANT NOTES: The instructions must be observed, irrespective of possible information given in any other way, including during demonstrations. It is the operator's responsibility to ensure that the product is suitable for the application in question. The Company is not liable for possible damage, including to third parties, deriving from failure to observe the instructions or inappropriate use of the product and in any case within the limits of the value of the products supplied.

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